HEBU medical GmbH  
Badstraße 8  
78532 Tuttlingen  
Germany

<b>D</b>	Gebrauchsanweisung für HF- Zubehör.....	Seite	<b>3</b>
<b>GB</b>	Instructions for HF accessories.....	page	<b>9</b>
<b>F</b>	Mode d'emploi pour accessoires HF.....	page	<b>13</b>
<b>ES</b>	Instrucciones para el uso de accesorios AF.....	página	<b>18</b>
<b>I</b>	Istruzioni d'uso per prodotti ad alta frequenza.....	pagina	<b>24</b>

### **Verwendung**

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## Medical Device Directive (Europe)

This device complies with the requirements of Directive 93/ 42 EEC concerning medical devices

### Use

HF-cables connect electrode handles, electrodes and instruments for monopolar or bipolar coagulation to HF units. When used as intended, these products are more or less subject to wear, depending on the intensity of use. This kind of wear is caused by technical factors and cannot be avoided. Therefore carry out a thorough visual examination and a function test prior to each application of these products. If a new product has externally visible defects or does not operate according to the instructions, contact the manufacturer or his responsible representative immediately.

### Operating

Thoroughly examine all products for visible irregularities. Do not use cables with a brittle or defective insulation. In case of doubt check operation by means of a suitable testing device.

### Connection to HF unit in monopolar mode

First fix the neutral electrode ("return plate") to the patient, preferably on the upper arm or on the thigh. Ensure that the skin is free from hair and grease. If a reusable neutral electrode (return plate) is used, do not apply conductive gel. The current paths inside the patient's body should be short and must run diagonally. Current paths must never run in a transverse direction through the body or across the thorax. The patient must be insulated against all electrically conductive parts. Ground the operating table. Place the patient on a dry electrically insulating layer. Prevent skin areas touching each other. Place dry gauze in between. Then connect the neutral electrode, the foot switch and the "active" HF cable to the HF unit. In order to avoid interference with camera system, do not run HF cables directly in line with camera cables. Never place HF cables directly on the patient's skin. Never lay HF cables in loops. Next, connect the instrument or handle with coagulation electrode, and switch on the HF unit. The electrode with the proper shaft diameter (4 mm or 2.4 mm) must be firmly attached to the handle tip. For handles with finger switches:

Yellow switch = actuation of cutting current

Blue switch = actuation of tissue coagulation

Always refer to the instruction manual of the HF unit and to the general information it provides relating to electrosurgical procedures. All monopolar products can be operated at voltages of up to 8500 V<sub>p-p</sub> (monopolar forceps up to 5500 V<sub>p-p</sub>).

### Connection to HF unit in bipolar mode

In bipolar HF surgery the coagulation is restricted to the area between the two electrodes of the bipolar instrument. Consequently, a neutral electrode is necessary. Otherwise, follow the recommendations regarding patient preparation in monopolar operating mode and the corresponding instructions for the HF unit. All bipolar products (marked with an "\*" in this instruction ) can be operated at voltages of up to 2000 V<sub>p-p</sub> (bipolar forceps up to max. 1100 V<sub>p-p</sub>).

When using the bipolar "Auto start" mode, the cable used should not be longer than 4.5 m. Alternatively use a twin cable with max. length of 5 m.

## Warnings

The products may only be used in combination with HEBU accessories and only by clinically instructed and qualified personnel. Electrodes must not be twisted, as this may cause damage to the material and the electrode might break. **Power output of the HF unit:** Always select the lowest possible output of the HF unit for any procedure. If the coagulation capacity of the electrode is below normal, do not increase the output of the HF unit without a thorough preliminary check. In particular, the following must be checked: correct attachment of all HF cables and plugs, correct activation of current by the finger - or footswitches, undamaged insulation of the HF cables and instrument, and check that the distal end of the electrode is dean and undamaged. Before operating, check that the signal transmission of the monitors works without interference (e.g. without noise); this check is made by pressing the yellow button (CUT) and the blue button (COAG). Always unplug the HF-cable by holding on the connector. **Never pull the cable itself**, as this may lead to defects. Cables with defective /brittle insulation or with a broken electrical wire (due to severe bending or even cause a fire. Active HF instruments and HF handles must not be placed on the patient in order to avoid an injuries in case of involuntary HF activation or activated "Auto Start" mode.

## Caution

For cleaning the active electrode, you must deactivate the "Auto Start" mode (only in bipolar applications) or disconnect the instrument from the generator. Directly after activation the tips could be hot. This may lead to burns.

**Endogenous burns** are those caused by high current density in the patient's tissue. Possible reasons are: The patient is inadvertently positioned in contact with electrically conductive parts. Direct contact between skin areas and HF cables may lead to electrical capacitance which in turn may cause burns.

**Exogenous burns** are those caused by heat of ignited fluids or gases. They may also be caused by explosions. Possible reasons include: Ignition of skin cleaning agents and disinfectants, ignition or narcotic gases etc.

**Cardiac pacemakers** may be damaged by electrosurgical current. Consult a cardiologist before operating. Never use electrosurgical current on outpatients fitted with pacemakers.

## Cleaning, disinfection and sterilisation

Caution: Products are supplied in an unsterile state and must be cleaned and sterilised before use. Do not clean HF cables, adapters or electrode handles in an ultrasonic bath. Do not sterilise HF cables, adapters or electrode handles and electrodes in hot air sterilisers. After use, wipe all surfaces with an alcoholic surface disinfectant immediately. The HF cable or handle may also be immersed in a combined cleaning and disinfection solution; in case, the service life may then be reduced due to oxidation of the contacts and crystallisation in the plugs. If a washing machine is used for cleaning: max 95 °C. Please observe the chemical manufacturer's instructions. Then rinse cable, connectors, and handle or electrode thoroughly with sterilised water. Steam-sterilise products at 134°C / 20 min.

## Repairs

Defective HF cables or electrode handles may not be repaired. Always replace a defective part with a new one.

134 °C  
Autoclave

### **Storage**

Temperature: -20 °C - +50°C

Relative humidity: 0 -75%, not condensing

Atmospheric pressure: 500 -1600 mbar

Products must be protected from direct sunlight in storage. The storage of the products in their original packaging until their first use is recommended.

---

### **Additional information for the use of argon accessories**

#### **Intended use**

The argon surgical handle is a re-usable electrosurgical handle with gas feed. The handle, with the monopolar rigid argon electrodes, can be used in conjunction with HF generators for peak voltages of max 4.200 Vp. It is used in argon gas assisted HF surgery . It is suitable for the fitting of monopolar, rigid argon electrodes, which are used for argon assisted coagulation or for argon gas assisted electrosurgical cutting. The cutting and coagulation currents can be controlled on the HF surgical unit by means of two finger switches.

Fundamentally, the argon surgical handle can also be operated by means of a foot switch. These products may only be used in medical establishments by trained personnel. Read the instruction manual carefully before using the products.

#### **Instrument preparation**

Mechanical precleaning of argon knife and needle electrodes can be carried out in the same way as with conventional surgical electrodes. As liquid can enter the gas feed pipe of the handle during the cleaning process, the lumen of the gas feed pipe should be blown free with compressed air, max. pressure 4 bar, after cleaning. For this purpose, the compressed air fitting is placed loosely on the Luer Lock connector of the argon surgical handle (do not blow out from distal to proximal).

The argon surgical handle is sterilised in the autoclave at up +134°C for 20 minutes.

#### **Do not sterilise in hot air!**

Gas sterilisation is not recommended, as the airing times have not been evaluated for this product.

#### **Connecting the argon surgical handle**

It must be ensured first of all that the HF generator and the argon gas control unit are ready for operation. The gas bottle valve must be open and there must be an adequate supply of gas.

1. Remove the handle and electrodes from the container in a sterile manner.

2. Visually inspect the instrument for damage to the insulation. Do not use damaged instruments! HEBU warns against repairing damaged areas. The instrument is operated at high coagulation voltages!
3. Connect 3-pin plug to the HF surgical unit.
4. Plug the required argon electrode into the electrode socket on the handle. Make sure, that the plastic hexagonal section of the electrode shaft is fully inserted into the handle.
5. Make the argon gas connection to the argon control unit. To do this, connect the male Luer-Lock connector of the handle to the female Luer-Lock gas outlet of the argon control unit.
6. Activate the purge function of the argon gas control unit in order to flood the gas feed pipe of the handle with argon gas.
7. Operate the foot switch and/or finger switches of the argon surgical handle one after the other and check the operation of the system (activation of the argon gas control unit and HF generator).

#### **Safety information**

Never place the gas outlet nozzle of the rigid argon electrode directly onto the tissue during gas activation. **Risk of gas emphysema**  
The gas outlet nozzle should be moved and activated vertically above the tissue. An angle of ca. 45° to the tissue has been shown to be beneficial. Do not blow argon in into the vascular system ➡ **Risk of embolism!**

With laparoscopic applications, the argon gas flow increases the intra-abdominal pressure. The use of electronic pressure-relief controlled CO<sub>2</sub> insufflators is recommended. Monitor the intra-abdominal pressure during activation and, if necessary, relieve the pressure by means of the trochar.

With endoscopic applications using video endoscopes, the argon plasma beam must not be directed onto the camera chip.

High HF voltages are required for gas ionisation: it must be ensured that the electrical insulation of cables and applicators is undamaged.

**The argon gas assisted application is a monopolar type of HF surgical application; the safety regulations for monopolar high frequency surgery must be observed .**

Argon with a quality rating of 4.8 is free from germs in the gas bottle as a result of the manufacturing process. In order to guarantee freedom from germs, the bottle valve and the gas outlet on the gas supply unit should be disinfected before connecting.

### **Application information**

Rigid argon coagulation electrodes are operated exclusively with spray coagulation current.

#### **Always operate argon coagulation electrodes by means of the blue buttons!**

Basically, argon knife and needle electrodes (cutting electrodes) can be used with argon gas assistance with all type of monopolar cutting and spray coagulation current. Unmodulated cutting current is recommended for gas-assisted cutting.

In order to "fire" the argon plasma, the argon electrode is moved within ca. 1 cm of the tissue. During the argon-assisted coagulation process (beaming), this distance can be increased to ca 2-2,5 cm before the plasma extinguishes.

### **Open surgical application**

The argon surgical handle allows a gas feed in the range 0-9.5 l/min. The gas flow is selected on argon control unit depending on the selected argon electrode and the indication. Flow rates of 2-4 l/min have been shown to be beneficial for argon electrodes in open surgical applications. In this case, the output power of the HF generator in spray mode is ca. 40 W.

When cutting, an output power of ca. 60 W has been shown to be beneficial for argon needle electrodes and a power of ca. 90 W for argon knife electrodes. In the cutting mode, the gas flow is ca. 2-4 l/min.

### **Laparoscopic application**

The trochar valve must be open during argon application in order to ensure that a critical intra-abdominal pressure is not exceeded. If an insufflator pressure has fallen below the critical value once more.

Whenever the intra-abdominal pressure is monitored by an insufflator, an independent, continuous monitoring of the intra-abdominal pressure by the operating team is required. The use of a pressure-relief controlled insufflator is recommended.

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