# Containersysteme

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# Contents

|       |  |    |
|-------|--|----|
| 1     | Symbols .....                            | 18 |
| 2     | Introduction .....                       | 18 |
| 3     | validity area .....                      | 18 |
| 4     | Warning .....                            | 20 |
| 5     | Scope .....                              | 21 |
| 5.1   | Intended Use .....                       | 21 |
| 5.2   | Contraindications .....                  | 21 |
| 6     | Preparation .....                        | 21 |
| 6.1   | Recommended sterilisation method .....   | 21 |
| 7     | Handling .....                           | 22 |
| 7.1   | General .....                            | 22 |
| 7.2   | Preparation before cleaning .....        | 24 |
| 7.3   | First use .....                          | 24 |
| 7.4   | Cleaning and disinfection .....          | 24 |
| 7.4.1 | Manual cleaning .....                    | 25 |
| 7.4.2 | Mechanical cleaning .....                | 25 |
| 7.4.3 | Recommended cleaning method .....        | 25 |
| 7.5   | Testing .....                            | 25 |
| 7.6   | Filter change .....                      | 26 |
| 8     | Loading .....                            | 27 |
| 9     | Placement in the steriliser .....        | 28 |
| 10    | Processing .....                         | 28 |
| 11    | Storage .....                            | 28 |
| 12    | Maintenance / Repair .....               | 29 |
| 13    | Materials .....                          | 29 |
| 14    | Warranty .....                           | 29 |
| 15    | Manufacturer and servicing address ..... | 29 |

## 1 Symbols

| Symbol  | Definition   |
|---|--------------|
|  | CE marking   |
|  | Attention    |
|  | Manufacturer |

## 2 Introduction

With the purchase of this sterilisation container you have chosen a high quality product. The correct handling and use are described in the following.

In order to keep risks and unnecessary burdening as low as possible for the patient, the user and third parties, we request that you carefully look through these instructions for use and keep them for further reference.

Our products are exclusively intended for professional use by appropriately trained and qualified personnel and may only be acquired by them.

## 3 validity area

The operating manual is available for the following articles:

| HEBU No.   | Article description   |
|------------|---|
| HE 3110-11 | 1/1 spare box alone without filter, 580x280x110mm               |
| HE 3110-16 | 1/1 spare box alone without filter, 580x280x160mm               |
| HE 3110-21 | 1/1 spare box alone without filter, 580x280x210mm               |
| HE 3110-26 | 1/1 spare box alone without filter, 580x280x260cm               |
| HE 3210-11 | 1/1 spare box alone with 2 filters in the bottom, 580x280x110mm |
| HE 3210-16 | 1/1 spare box alone with 2 filters in the bottom, 580x280x160mm |
| HE 3210-21 | 1/1 spare box alone with 2 filters in the bottom, 580x280x210mm |
| HE 3210-26 | 1/1 spare box alone with 2 filters in the bottom, 580x280x260mm |
| HE 3310SI  | 1/1 spare lid alone with 2 filters, silver, 580x280mm           |
| HE 3310SW  | 1/1 spare lid alone with 2 filters, black, 580x280mm            |
| HE 3310BL  | 1/1 spare lid alone with 2 filters, blue, 580x280mm             |
| HE 3310GO  | 1/1 spare lid alone with 2 filters, gold, 580x280mm             |

| HEBU No.   | Article description  |
|------------|--|
| HE 3310GR  | 1/1 spare lid alone with 2 filters, green, 580x280mm           |
| HE 3310RO  | 1/1 spare lid alone with 2 filters, red, 580x280mm             |
| HE 3120-11 | 3/4 spare box alone without filter, 465x280x110mm              |
| HE 3120-16 | 3/4 spare box alone without filter, 465x280x160mm              |
| HE 3120-21 | 3/4 spare box alone without filter, 465x280x210mm              |
| HE 3220-11 | 3/4 spare box alone with 1 filter in the bottom, 465x280x110mm |
| HE 3220-16 | 3/4 spare box alone with 1 filter in the bottom, 465x280x160mm |
| HE 3220-21 | 3/4 spare box alone with 1 filter in the bottom, 465x280x210mm |
| HE 3320BL  | 3/4 spare lid alone with 1 filter, blue, 465x280mm             |
| HE 3320GO  | 3/4 spare lid alone with 1 filter, gold, 465x280mm             |
| HE 3320GR  | 3/4 spare lid alone with 1 filter, green, 465x280mm            |
| HE 3320RO  | 3/4 spare lid alone with 1 filter, red, 465x280mm              |
| HE 3320SI  | 3/4 spare lid alone with 1 filter, silver, 465x280mm           |
| HE 3320SW  | 3/4 spare lid alone with 1 filter, black, 465x280mm            |
| HE 3130-11 | 1/2 spare box alone without filter, 285x280x110mm              |
| HE 3130-16 | 1/2 spare box alone without filter, 285x280x160mm              |
| HE 3130-21 | 1/2 spare box alone without filter, 285x280x210mm              |
| HE 3130-26 | 1/2 spare box alone without filter, 285x280x260mm              |
| HE 3230-11 | 1/2 spare box alone with 1 filter in the bottom, 285x280x110mm |
| HE 3230-16 | 1/2 spare box alone with 1 filter in the bottom, 285x280x160mm |
| HE 3230-21 | 1/2 spare box alone with 1 filter in the bottom, 285x280x210mm |
| HE 3230-26 | 1/2 spare box alone with 1 filter in the bottom, 285x280x260mm |
| HE 3330SI  | 1/2 spare lid alone with 1 filter, silver, 285x280mm           |
| HE 3330BL  | 1/2 spare lid alone with 1 filter, blue, 285x280mm             |
| HE 3330GO  | 1/2 spare lid alone with 1 filter, gold, 285x280mm             |
| HE 3330GR  | 1/2 spare lid alone with 1 filter, green, 285x280mm            |
| HE 3330RO  | 1/2 spare lid alone with 1 filter, red, 285x280mm              |
| HE 3330SW  | 1/2 spare lid alone with 1 filter, black, 285x280mm            |
| HE3530BL   | 1/2 spare lid alone without filter, blue, 285x280mm            |

| HEBU No. | Article description                                   |
|----------|---|
| HE3530GO | 1/2 spare lid alone without filter, gold, 285x280mm   |
| HE3530GR | 1/2 spare lid alone without filter, green, 285x280mm  |
| HE3530RO | 1/2 spare lid alone without filter, rot, 285x280mm    |
| HE3530SI | 1/2 spare lid alone without filter, silver, 285x280mm |
| HE3530SW | 1/2 spare lid alone without filter, black, 285x280mm  |
| HE3510BL | 1/1 spare lid alone without filter, blue, 580x280mm   |
| HE3510GO | 1/1 spare lid alone without filter, gold, 580x280mm   |
| HE3510GR | 1/1 spare lid alone without filter, green, 580x280mm  |
| HE3510RO | 1/1 spare lid alone without filter, rot, 580x280mm    |
| HE3510SI | 1/1 spare lid alone without filter, silver, 580x280mm |
| HE3510SW | 1/1 spare lid alone without filter, black, 580x280mm  |

## 4 Warning

|   |   |
|---|---|
|  | Sterilization containers are delivered in a non-sterile condition and must be cleaned, disinfected and sterilized prior to their initial use. |
|  | The use of faulty instruments is in principle forbidden and they have to go through the whole cleaning process before return.                 |
|  | Please observe the additional information enclosed with the products.   |
|  | Remove all protective sleeves and films prior to first using or preparation for use.  |
|  | The safe combination of different must be reviewed prior to clinical application by the user.   |
|  | Avoid improper throwing or dropping of the products.  |
|  | To prevent all contact corrosion, products with damaged surfaces must be separated immediately.   |
|  | If used on patients with Creutzfeldt-Jakob disease or HIV infection, we are unable to accept any responsibility for reuse.                    |

## 5 Scope

The HEBUmedical Sterilization Container System is a sterile packaging system for medical instruments and textiles that utilizes an established filter technology. It is a reusable device and features an assortment of sizes and configurations that provide an effective packaging method for sterilization, storage and transportation of surgical instrument by healthcare providers. This container system is intended for use in pre-vacuum Steam Sterilizers.

It is intended to allow sterilisation of the enclosed device and also maintain sterility of the enclosed device until used.

All Containers are available with perforated and non-perforated bottom and perforated lids.

All containers use suitable disposable paper filters, which must be replaced before each sterilisation.

The use of container seals takes place in the Central Sterile Supply Department (CSSD).

### 5.1 Intended Use

Sterilisation containers and baskets:

A product used to store surgical instruments, usually in sets, electric drills, saws and their guides but also other instruments during sterilization and subsequent storage. The container is permeable so that any moisture can evaporate after sterilization; while the contents are protected from contamination. It offers possibilities such as fixing the expiry date, sterile filters and stacking options. It is a reusable product.

### 5.2 Contraindications

The following methods are not suitable for the containers:

- Hot-air sterilization, gravity or circulation procedures
- formaldehyde or ethylene oxide sterilization
- substitute procedures for the sterilization of thermolabile products such as plasma sterilization or peroxide sterilization

## 6 Preperation

### 6.1 Recommended sterilisation method

In order to ensure sterile safety, tests were carried out by an independent and accredited test laboratory. The purpose of these trials was to validate a sterilisation process for the reusable HEBUmedical sterilisation container with steam.

The HEBUmedical Container System was validated with the following sterilisation parameters. Each facility may need to run internal testing to determine if adjustments are necessary for their facility.

|               |  |
|---------------|--|
| method:       | three times pre-vacuum steam sterilization                             |
| temperature:  | 132 °C (270 °F)  |
| holding time: | 4 minutes  |
| Drying time:  | 10 minutes   |
| loading:      | standard surgical instruments (scissors, clamps, forceps) and textiles |

The validation considered the following Containers of the HEBUmedical Container System:

- Full-size container,
- ¾-size container,
- ½-size container

**An application of gas sterilization is not suitable because it is carried out with formaldehyde or ethyleneoxid. The containers are not made for it.**

## 7 Handling


### 7.1 General

The HEBUmedical container is made of aluminum alloy with an anodised oxide surface which prevents corrosion. They guarantee, due to their high thermal conductivity, an optimal drying phase. Abrasive cleaners, metal brushes or abrasive cleaning pads can cause permanent damage to the container surface and therefore must not be used. Warranty exclusions will be the result in case these instructions are not followed.

**The product and accessories must be used only by qualified or trained and experienced personnel, in order to prevent damage to the containers, closing devices, seals and sterile filters.**


All HEBU-containers can be stacked, whereby space end storage problems are solved. The sterilisation containers are also available with coloured lids. Allocation to various medical disciplines and specialist departments can thus be simplified using different colours.

The steam can circulate optimally by the perforated filter compartment in the lid. Moreover, the filter compartment is protected optimally against any impact by the robust protective cover. An optimal and barrier-free steam sterilization is still guaranteed.

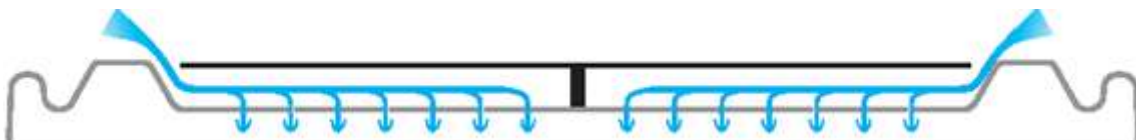**Fig. 1: Steam in the perforated filter compartment in the lid**

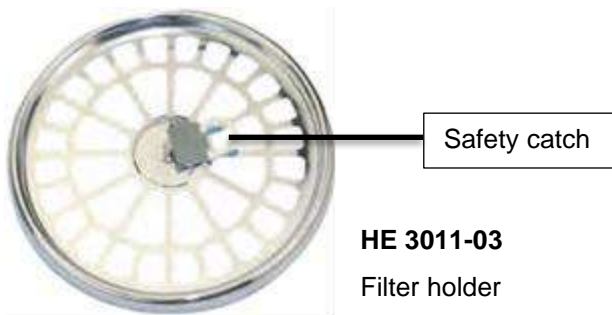
A silicon seal in the rim of the lid guarantees complete leak-proof closure during and after sterilization. The filter holder, with an integrated silicon seal, ensures solid end firm positioning of the filter (germ sealed). The safety catch of the filter holder allows the filter to be quickly and easily changed.

HEBU-containers have the possibility to be marked in several ways. By simple insertion of the labels in a specified slot, which is equipped with a safety spring, each container can be individually marked. There are paper labels with indicator and Aluminum labels in different colors with or without inscription.

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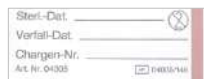|  | Without inscription | With inscription |
|--|---------------------|------------------|
|  | HE 3010-05BL        | HE 3011-05BL     |
|  | HE 3010-05GO        | HE 3011-05GO     |
|  | HE 3010-05GR        | HE 3011-05GR     |
|  | HE 3010-05RO        | HE 3011-05RO     |
|  | HE 3010-05SI        | HE 3011-05SI     |
|  | HE 3010-05SW        | HE 3011-05SW     |

To prevent the container from unauthorized opening, a security seal can be fixed on the opening handle which breaks as soon as the container is opened. Only an intact security seal ensures that the sterilisation container has not been opened without permission.

**HE 3000-00**  
Plastic Security Seal

## 7.2 Preparation before cleaning

1. separate lid and bottom
2. Remove the devices from the inside of the container (baskets, instruments, ...)
3. take off the filter holders inside the lid and, if existent, in the bottom (in case of perforated bottom containers)
4. throw away the disposable filter(s)
5. remove disposable locks and indicators

**All paper-filters are for single use only and should be thrown away after each processing-cycle**


Paper Filters

HE 3010-01

HE 3040-01

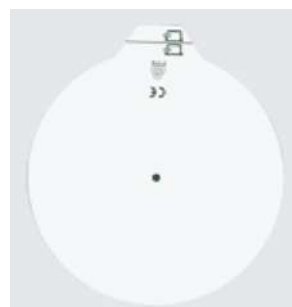HE 3050-01

PTFE Permanent Filter

HE 3010-06

HE 3040-06

HE 3050-06

## 7.3 First use

- The new container needs proper cleaning before first use.
- The container has come to terms with a validated, automated cleaning and disinfecting program.
- In the machine to use there should be a neutral detergent (pH 7).
- After processing in the cleaning and disinfection equipment, we recommend the sterilization of the container in 134°C (EU) or 132°C (USA).
- We also recommend the maintenance of all moving parts on the container with instrument oil. When using a paper filter, a suitable new filter must be used after cleaning (see 7.6 Filter change).  
PTFE Permanent filters do not need to be changed after first cleaning.

## 7.4 Cleaning and disinfection

**Note:** Improper cleaning and disinfection can lead to corrosion and stress cracks. Therefore, follow the specifications the cleaning and disinfecting agent's manufacturer recommends. Detergents must be alkaline-, sodium- and carbonate-free, of neutral pH (7) and mild. Only fully deionised water (quality according to EN 285 Annex B) is recommended for reprocessing the product.

- Before every use, the container has to be cleaned and disinfected.
- Removal containers have to be cleaned and disinfected each time after use.

### 7.4.1 Manual cleaning

- Mild detergents or neutral cleansers that are specially recommended for anodised aluminum by the manufacturer are to be used in combination with a soft sponge and water.
- After cleaning, careful rinsing and manual drying is necessary.
- Do not use metal brushes or scouring agents.

### 7.4.2 Mechanical cleaning

- Mechanical cleaning of the containers is only recommended if the washer/desinfector has a special programme for aluminum containers. When mechanically cleaning the containers, it is absolutely essential to observe the specifications of the washer/desinfector manufacturers and their instructions for the cleaning of containers.
- Only use neutral cleaners or neutral cleaners and disinfectants for cleaning. Do not use any cleaning solutions containing soda or caustic soda
- No additional use of acidic neutralizers
- Use neutral cleaners only with aluminum containers, which have been expressly approved by the manufacturer for the cleaning of aluminum containers. The exact dose of the cleaner is to be used as specified by the manufacturing company. With an optimisation of the program these products are also suitable for cleaning surgical instruments
- Fully demineralised water has to be used for final rinsing, as the salts in the water during the subsequent sterilisation can also lead to spotting
- The container lid and bottom need to be cleaned separated from each other.
- The container bottom needs to be placed upside down in the washer/desinfector to avoid the collection of water.
- The inside of the container lid should face the machine-bottom with the latching mechanism folded into the inner part of the lid.
- All component parts of the container (bottom, lid, filterholders...) should be disassembled and placed in the washer/desinfector baskets that are specially designed for containers and accessories.
- After the washing/desinfection program is finished, the container and accessories need to be dried with a soft dry cloth or by air.
- Contaminations that cannot be removed in a normal cleaning cycle independent of the process (sticky labels, indicator strips, writing), can be removed with anodic cleaners. After this special treatment, the containers should be cleaned as usual

### 7.4.3 Recommended cleaning method

We recommend the following cleaning method which was validated by HEBUmedical in a suitable cleaning and disinfection unit (Miele PG 8528) with the following settings:

- 1 min pre-wash with cold water
- 3 minutes Mediclean clean with 0.5% (Dr. Weigert) at 45 ° C
- neutralization with demineralized water

## 7.5 Testing

The sterilisation containers have to be checked every time before use to ensure their correct functionality.

If any damage of closing device, seals, filter holders, filters or any bent and dented parts occur, these container parts need to be taken out of circulation and must be returned for repair. Do not use any damaged sterilisation containers.

## 7.6 Filter change

For removing the filter holder, the safety catch must be pushed upwards and then the holder must be pulled in the direction of the arrow (see Fig. 2).

Fig. 2: Filter holder with a paper filter

After placing the filters over the perforated areas on the inside of the lid and (if applicable) on the bottom of the container, the lock have to be pressed down until the filter holder snap into position. HEBUmedical lids should only be assembled with HEBUmedical filter holders.

- The disposable paper filters have to be exchanged before every new sterilisation cycle.
- Only HEBUmedical filters can ensure suitability and exactness of fit.
- Guarantee can only be accepted when original HEBUmedical filters have been used

**Only original HEBUmedical component parts such as lids, bottoms, filters, seals and filter holders should be combined with each other so that the container sealing and function as a germ barrier is not compromised. All filters or seals made by other manufacturers must be approved by HEBUmedical before used with the HEBUmedical container-system. Otherwise warranty exclusion will be the result.**

## 8 Loading

The overall weight for loading containers should not exceed the following recommended loads, as otherwise satisfactory sterilisation of sterile materials cannot be ensured.

| Model                      | Dimension   | Maximum Recommended Load in kg |          |
|----------------------------|-------------|--------------------------------|----------|
|                            |             | Instruments                    | Textiles |
| 1/1 (Full-) Size Container | 580x280x110 | 3,8                            | 3        |
|                            | 580x280x135 | 4,8                            | 3,8      |
|                            | 580x280x160 | 5,8                            | 4,6      |
|                            | 580x280x210 | 7,7                            | 6,1      |
|                            | 580x280x260 | 10                             | 8        |
| ¾ Size-Container           | 465x280x110 | 3,1                            | 2,5      |
|                            | 465x280x135 | 3,8                            | 3,1      |
|                            | 465x280x160 | 4,6                            | 3,7      |
|                            | 465x280x210 | 5,9                            | 4,7      |
| ½ Size-Container           | 285x280x110 | 1,9                            | 1,5      |
|                            | 285x280x135 | 2,3                            | 1,8      |
|                            | 285x280x160 | 2,8                            | 2,2      |
|                            | 285x280x210 | 3,8                            | 3        |
|                            | 285x280x260 | 4,9                            | 3,9      |

When loading with textiles, pay attention that the folded textile or laundry articles are positioned vertically. In the case of a full container, it should be possible to push an outstretched hand between the pieces of laundry without effort

**The sterilisation of various container loads and packaging configurations has to be validated by the responsible specialist hygiene personnel according to ISO 11134.**

**Complex instruments like Endoscopes, instruments with a lumen, compressed air-driven instruments or power systems and instruments with cannulae are to be prepared according to the manufacturers' instructions for sterilisation and to be sterilised accordingly. Small baskets, trays, other types of accessories, especially with cover or lids, should only be used with the sterilisation container, if the sterilisation container has been specifically designed and tested for that purpose. Please see details about HEBUmedical frames, baskets, and metal sterilising trays in our product catalogue.**

**When using non-absorbent tray liners (e.g., plastic/silicone-fingered organising mats) for the containers this can cause condensate to pool. Instead of this use moisture absorbing mats if necessary.**

Always attach the lid to the bottom via the latching mechanism before placing the container in the steriliser. Otherwise the content of the container will become unsterile as soon as the steriliser door will be open.

## 9 Placement in the steriliser

The sterilisation containers are constructed so that they can be used in any conventional large steam steriliser.

Pay attention that heavy containers are placed at the bottom of the sterilisation chamber.

Thanks to the way they are constructed, the sterilisation containers can be stacked safely and without a problem on top of one another during sterilisation without slipping out of place.

Only stack containers in high-vacuum cycles and do not stack them higher than 16-18" to allow an effective air removal and steam penetration. The loading instructions of the steriliser manufacturer should be observed.

**! Never wrap the container in any kind of outer packaging. Never cover the perforated area of the container with any kind of foil packaging or similar during sterilisation, because this will block the air and steam flow through the perforation. The result will be a vacuum damage due to insufficient pressure venting and the container content will not be sterilised. During loading and unloading of the steriliser and during transport, the sterile container must always be carried by the handles and never by the lid.**

## 10 Processing

- Follow the time and temperature recommendations of the steriliser manufacturer for every chosen sterilisation cycle.
- After sterilisation leave sterilizer door open at 6" for approximately 15 minutes to aid drying (see recommendation of steriliser manufacturer).
- To minimize condensate inside the container, leave container on container carts until cool enough to handle.
- Always check if the sterilisation has been successful by observing indicators and the container latch (must be closed).
- warranties will only be accepted by using original HEBUmedical filter.

## 11 Storage

Sterile material maintains its sterility from several weeks to up to 6 months under normal aseptic conditions, in unopened sterilisation containers and with undamaged sterile filters. Usually the storage period depends on the storage conditions and must be defined by the responsible specialist hygiene personnel.

In the case of particularly high demands on sterility, for example, shorter storage times or additional packaging should be used.

Recommended storage conditions:

- Temperature 15 – 26 ° C
- Air humidity 30 – 50%
- Air pressure, normal atmospheric pressure

Various container loads, storage periods and storage conditions have to be validated by the responsible specialist hygiene personnel.

## 12 Maintenance / Repair

- The useful life of undamaged cover seals is at least 500 sterilisation cycles. The cover seals then have to be replaced.
- Cover seals have to be exchanged if they show signs of damage.
- Cover seals should not be treated with spray, oil or solvents. For cleaning and maintenance simply wipe off occasionally with a damp cloth.
- If any damage on the container or its parts is noticed the container needs to be taken out of circulation immediately and returned for repair.
- Qualified persons only may carry out maintenance and repair of sterilisation containers. Do not attempt to carry out repairs on the cover seals or closing devices yourself, in order not to jeopardise the safe function of the container.
- Sterilisation containers can be sent to HEBUmedical or one of the authorised repair services for maintenance and repairs.
- Replacement parts can be obtained from HEBUmedical

## 13 Materials

Sterilisation containers are made of anodised aluminum alloy and their accessories of surgical stainless steel.

## 14 Servicable Life

The end of serviceable life tends to be determined by mechanical wear and tear and damage rather than frequent use of the container for its intended purpose and can be identified by the required control of function before reuse (see 7.5 Testing).

The containers are designed for a service life of approx. 10 years.

## 15 Warranty

These sterilisation containers have been manufactured from high quality materials and have been subjected to quality control checks before release to the market. Nevertheless, revert to the above-mentioned address in case any errors should arise.

The warranty shall lapse if companies who are not authorised for repairs by HEBUmedical carry out repairs.

**THE HEBUmedical COMPANY ACCEPTS NO LIABILITY IN DEMONSTRABLE CASES OF VIOLATION OF THIS CUSTOMER INFORMATION.**

## 16 Manufacturer and servicing address

HEBUmedical GmbH  
Badstraße 8  
78532 Tuttlingen / Germany  
Tel. +49 7461 94 71 - 0  
Fax +49 7461 94 71 - 22  
eMail: [service@HEBUmedical.de](mailto:service@HEBUmedical.de)  
Web: [www.HEBUmedical.de](http://www.HEBUmedical.de)

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