# HF-Neutralelektroden, wiederverwendbar / HF return plates, reusable

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Seite 3

DEUTSCH

**Operating Manual**

Seite 17

ENGLISH

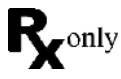
# Inhalt

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# Contents

1	Symbol descriptions .....	18
2	Introduction .....	19
3	Scope .....	19
3.1	Intended Use .....	19
3.2	Contraindication .....	19
4	Handling .....	19
5	Commissioning the unit .....	20
5.1	Neutral electrode settings .....	20
5.2	Application information .....	20
5.3	Areas for attachment of the neutral electrode .....	21
5.4	Connection to HF unit in monopolar mode .....	22
6	Warnings .....	22
7	Safety Remarks .....	23
8	Protective measures and warnings .....	23
9	Preparation .....	24
9.1	Reutilization restrictions .....	24
9.2	Information on instrument preparation .....	24
9.3	Preparation at the place of use .....	24
9.4	Ultrasound bath (optional) .....	24
9.5	Manual cleaning .....	25
9.6	Mechanical cleaning .....	25
9.7	Drying .....	27
10	Maintenance, inspection .....	27
10.1	Functional check .....	27
11	Sterilization .....	27
11.1	Packaging .....	28
12	Lifetime .....	28
13	Storage .....	28
14	Warranty/Repair .....	28
15	Servicing and manufacturer address .....	29

# 1 Symbol descriptions

Symbol	Definition
	CE-labelling
	Attention
	Validated Parameters
	Manufacturer
	Lot-description
	Reference code
	Medical device / FDA Prescription device
	Medical device
	Non sterile
	Keep away from sunlight
	Dry storage required
	Temperatur
	(Electronic) instrucion for use
	No intervention on patients with cardiac pacemakers.

## 2 Introduction

Our products are exclusively intended for professional use by appropriately trained and qualified personnel and may only be acquired by them.

You are now the owner of a high-quality product whose use and correct handling are described in the following. In order to minimize possible risks to patients and users, please observe these instructions carefully. Use, disinfection, cleaning and sterilization may only be performed by suitably trained specialist personnel.

## 3 Scope

The use of HF neutral electrodes takes place in HF surgery.

### 3.1 Intended Use

HF Neutral electrodes: A non-active conductor attached to a patient and connected to a high frequency electrosurgical generator to establish a circuit for the return of electrical current to the generator after the transmission of the current for performing electrosurgical procedures on the patient. It is attached to the patient where the largest surface can be covered as close to the surgical field as possible. It is a reusable article.

### 3.2 Contraindication

The HF neutral electrodes must not be used on damaged and injured skin.

## 4 Handling

The type of treatment must be determined in each individual case by the surgeon in cooperation with the internist and the anaesthetist.

For operational use in various surgical disciplines must be done by appropriately trained and qualified personnel.

## 5 Commissioning the unit

### 5.1 Neutral electrode settings

In the monopolar operating mode, a neutral electrode is required.

After attaching the neutral electrode to the patient, the HF unit recognizes the neutral electrode automatically (cf. illustration on the left and chapter **Fehler! Verweisquelle konnte nicht gefunden werden. Fehler! Verweisquelle konnte nicht gefunden werden.**). The HBS x-touch 200i recognizes neutral electrodes with divided and undivided contact surfaces.

**If the displayed neutral electrode type does not agree with the used neutral electrode type, this is indicative of poor adhesion of the neutral electrode on the patient, a defective neutral electrode or a defective cable!**

**Please note the following information when attaching the neutral electrode**

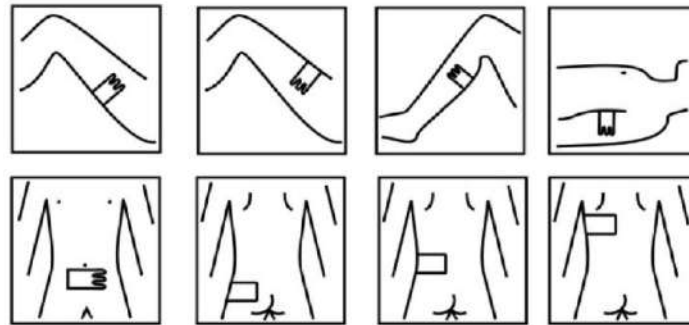
### 5.2 Application information

- The neutral electrode must be attached in such a way that its entire surface adheres to the patient's body. A rubber band provided for this purpose can be used to ensure better adhesion.
- When attaching the neutral electrode, ensure that the long side is pointing towards the operation field.
- After each change in the patient's position, check the correct adhesion of the electrode and the connecting cable!
- The electrode must not be attached to surfaces with a high amount of hair growth. If necessary, shave the relevant area before attaching.
- The electrode must not be wet or coiled round.
- Fluids between the patient's skin and the neutral electrode must be avoided.
- Do not apply any additional electrically conductive gel to the neutral electrode.
- When performing surgery on small children, special electrodes suitable for children or babies must be used.
- When removing the neutral electrode, never pull on the cable!
- Never under any circumstances carry out repairs on the neutral electrode!


Please note that rubber electrodes lose their conductive capability if active substances are eroded due to cleaning of the material. This type of electrode considerably increases the risk of burns. Consequently ensure that not only the unit but also the reusable neutral electrode is included in regular maintenance inspections.

### 5.3 Areas for attachment of the neutral electrode


- Attach the neutral electrode to clean, dry skin.
- The electrode must be attached close to the operation area, but with a minimum direct distance from the operating field of 20 cm.
- Attach over tissue with a good supply of blood without dips or curves in the skin, for instance on the upper arm or thigh. (cf. picture below)
- Do not attach the neutral electrode to projecting surfaces.
- Do not attach above bones, scars, cuts or scratches.
- The electrode must not be attached to surfaces with a high amount of hair growth. If necessary, shave the relevant area before attaching.
- Do not attach above particularly adipose areas such as the abdomen or buttocks.
- The neutral electrode must not be attached above implants.
- Electrodes of monitoring systems must be placed at a distance of at least 20 cm from the operation area and from the position of the attached neutral electrode
- Correct attachment of the neutral electrode in adult patients

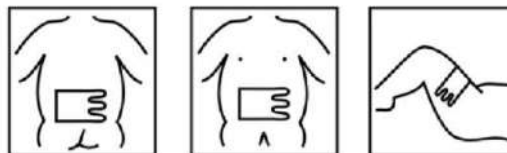- Correct attachment of the neutral electrode in children

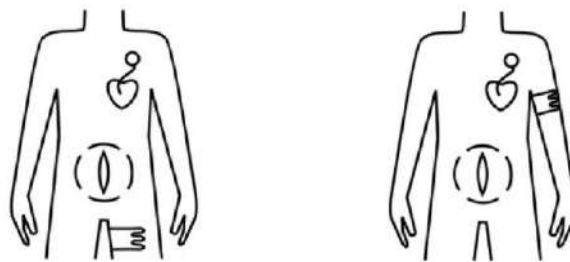- Attachment in patients with pacemakers  
**Note! Consult the treating cardiologist!**

**RIGHT**

**WRONG!**

## 5.4 Connection to HF unit in monopolar mode

First fix the neutral electrode ("return plate") to the patient, preferably on the upper arm or on the thigh. Ensure that the skin is free from hair and grease. If a reusable neutral electrode (return plate) is used, do not apply conductive gel. The current paths inside the patient's body should be short and must run diagonally. Current paths must never run in a transverse direction through the body or across the thorax. The patient must be insulated against all electrically conductive parts. Ground the operating table. Place the patient on a dry electrically insulating layer. Prevent skin areas touching each other. Place dry gauze in between. Then connect the neutral electrode, the foot switch and the "active" HF cable to the HF unit.

## 6 Warnings

The products may only be used in combination with HEBU accessories and only by clinically instructed and qualified personnel. Electrodes must not be twisted, as this may cause damage to the material and the electrode might break. Power output of the HF unit: Always select the lowest possible output of the HF unit for any procedure. If the coagulation capacity of the electrode is below normal, do not increase the output of the HF unit without a thorough preliminary check. In particular, the following must be checked: correct attachment of all HF cables and plugs, correct activation of current by the finger - or footswitches, undamaged insulation of the HF cables and instrument, and check that the distal end of the electrode is dean and undamaged. Before operating, check that the signal transmission of the monitors works without interference (e.g. without noise); this check is made by pressing the yellow button (CUT) and the blue button (COAG).

Always unplug the HF-cable by holding on the connector. **Never pull the cable itself**, as this may lead to defects. Cables with defective/brittle insulation or with a broken electrical wire (due to severe bending) might cause a fire. Active HF instruments and HF handles must not be placed on the patient in order to avoid injuries in case of involuntary HF activation or activated "Auto Start" mode.

**Endogenous burns** are those caused by high current density in the patient's tissue. Possible reasons are: The patient is inadvertently positioned in contact with electrically conductive parts. Direct contact between skin areas and HF cables may lead to electrical capacitance which in turn may cause burns.

**Exogenous burns** are those caused by heat of ignited fluids or gases. They may also be caused by explosions. Possible reasons include: Ignition of skin cleaning agents and disinfectants, ignition or narcotic gases etc.

**Cardiac pacemakers** may be damaged by electrosurgical current. Consult a cardiologist before operating. Never use electrosurgical current on outpatients fitted with pacemakers.

## 7 Safety Remarks

	Medical products are delivered in a non-sterile condition and must be cleaned, disinfected and sterilized prior to their initial use.
	A visual inspection of the instrument for damage and contamination must be carried out before each use!
	Avoid mechanical overstressing of the instrument beyond the structural design, this can lead to breakage and deformation!
	A visual inspection of the instrument for damage and contamination must be carried out before each use!
	The use of faulty instruments is in principle forbidden and they have to go through the whole cleaning process before return.
	Please observe the additional information enclosed with the products.
	Remove all protective sleeves and films prior to first using or preparation for use.
	The safe combination of different products or of products with implants must be reviewed prior to clinical application by the user.
	Avoid improper throwing or dropping of instruments
	To avoid any damage, HF neutral electrodes with a damaged surface must be removed immediately!
	If the products are used on patients with transmissible spongiform encephalopathy or HIV infection, we decline any responsibility for their reuse.
	Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## 8 Protective measures and warnings

	Use the greatest care when working with the neutral conductor!
	Prevent the penetration of fluids between the patient's body and the neutral electrode.
	A marked drop in power at a normal setting can be an indication of incorrect application of the neutral electrode.

## 9 Preparation

The person in charge of preparatory treatment is responsible for ensuring that the treatment is duly carried out using the relevant equipment, materials and personnel in the treatment facility and so achieves the desired result. This necessitates validation and routine monitoring of the process used. We urge you to take note of the national regulations dealing with instrument preparation.

### 9.1 Reutilization restrictions

Frequent repeat preparatory treatment has minimal effects on the product. The end of the product life is normally determined by wear and damage due to use

### 9.2 Information on instrument preparation

- Use cleaning and/or disinfection agents with a pH-value within 9-10. Please observe manufacturer instructions regarding dosage, exposure time and renewal of solutions.
- Do **not** use hard brushes (e.g. metal brushes or metal sponges) or coarse abrasive cleaners.
- Never leave instruments in cleaning or disinfection agents for longer than the specified time.
- Only used demineralized water for rinsing.
- Rinse and dry carefully through channels and pipes.
- Sensitive instruments must be cleaned in a storage or clamping fixture.
- Observe manufacturer instructions of cleaning – and sterilizing equipment.

### 9.3 Preparation at the place of use

Directly after using remove coarse dirt of the instruments and rinse out the working cannulas. Do not use fixing agents or hot water (> 40°C), as this results in residues becoming fixed and can affect the success of the subsequent cleaning operation

Dismantle and/or open instruments as far as possible. Within short time after use the instruments clean the instruments for reducing a drying of the residues.

This enables an easier cleaning. If instruments come into contact with corroding medicines or cleaning agents, wash these up with water immediately after use.

Longer drying times, e.g. for dry disposal are not validated and therefore not recommended.

The drying time during validation was 1 hour.

### 9.4 Ultrasound bath (optional)

All instruments must be opened, dismantled and any cavities rinsed through.

Place instruments in the screen basket in such a way that overlaps and contact between instruments are avoided. Add cleaning agent to the water and adjust the temperature of the solution in line with the cleaning agent manufacturer's instructions.

The cleaning in the ultrasound bath should be at 35-40 kHz, 5 minutes at least.

To validate cleaning in an ultrasonic bath, the test items were ultrasonically treated in Neodisher Mediclean forte 0,5 % for 5 minutes.

Subsequently rinse instruments including all cavities before cleaning and disinfection.

Medicine products which are possessing a bad ultrasound transmission, e.g. soft material are not usable for the ultrasound bath.

## 9.5 Manual cleaning

Since mechanical processes can be standardized, reproduced and therefore validated, mechanical cleaning/disinfection should be preferred to manual processes.

Manual cleaning and disinfection process is not validated and therefore needs to be validated additionally by the end user.

## 9.6 Mechanical cleaning

On the basis of international standards (EN ISO 15883) and national directives, only validated machine cleaning and disinfection methods may be used. For the mechanical cleaning we recommend a standard programme for surgical instruments, f.e. instruments from Miele.

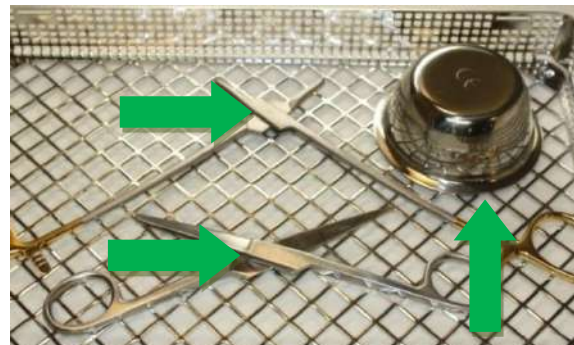
Only completely demineralized water should be used for cleaning, neutralisation and rinsing, in accordance with the „Guidance Compiled by the DGKH (Germany Society for Hospital Hygiene), DGSV (German Society for Sterile Supply) and AKI (Working Group on Instrument Reprocessing) for the Validation and Routine Monitoring of Automated Cleaning and Thermal Disinfection Processes for Medical Devices as well as Advice on Selecting Washer-Disinfectors“ (which refers to DIN EN ISO 15883-1 Point 6.4.2)

Flexible (complex) instruments with invisible surfaces have to be pre-cleaned manually before mechanical cleaning.



We recommend for all push shafts, -and pipe shafts instruments and instruments whose surfaces are on top of each others during the cleaning (f.e. bone forceps and gouge forceps) a manually pre-cleaning for an optimal cleaning result without residues

Observe the following by loading:

- Place the dismantled/opened instruments securely in the tray.
- Instruments with openings and gaps have to be faced down with the opened side so that they can be cleaned and no water of the cleaning process is collecting inside them. If available use balanced devices for rinsing

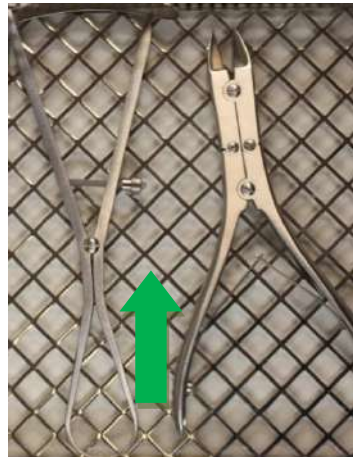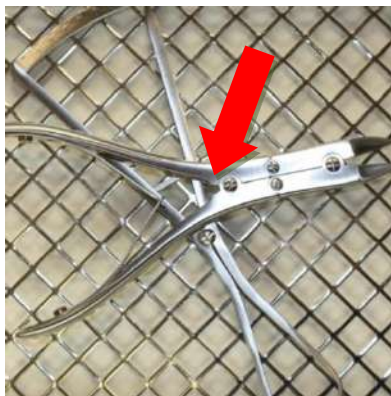- Do not overload trays, avoid creating any overlaps.

Preliminary rinsing (cold, if applicable fully demineralized water without additives) is followed by chemical. The chemical cleaning should take place at **40°C -60°C** for at **least 5 minutes**.

We recommend products with a **pH-value within 9-10**, e.g. Neodisher MediClean forte from Dr. Weigert. The cleaning agents used should be selected depending on the material and properties of the instruments and in accordance to national regulations: If there is a high chloride concentration in the water, pitting and tension crack corrosion can occur on the instruments. The occurrence of this type of corrosion is minimized by using alkaline cleaning agents and demineralized water. By adding an acid-based neutralization medium, the rinsing off of alkaline cleaning agent residues is facilitated during the first intermediate rinsing process (warm or cold water).

In order to prevent the formation of deposits, it is advisable to use neutral cleaners where the water quality is unfavourable. After the second intermediate rinsing process, thermal disinfection takes place.

The thermal disinfection should take place at temperatures of between **80 and 95°C**, with an **exposure time as outlined in EN ISO 15883**.

After the finishing of the programme take the good out of the machine because corrosion can arise if the instrument remains in the machine.

 Parameters used for the validation of preparation	
<b>Pre-rinsing</b>	1 minute with cold tap water
<b>Cleaning</b>	Temperature: 55 °C
	Soaking Time: 5 minutes (worst case)
	Neodischer Mediclean forte 0,4% (worst case)
<b>Neutralization</b>	Temperature: cold DI water
	Soaking Time: 2 minutes
	Neodisher Z 0,1%
<b>Post-rinsing</b>	2 minutes with cold DI water
<b>Disinfection</b>	Temperature: 90 °C (A <sub>0</sub> 3000)
	Soaking Time: 5 minutes

## 9.7 Drying

Ensure adequate drying by the cleaning and disinfection device or using other suitable measures.

**V** Drying was omitted in the validation (worst case condition).

## 10 Maintenance, inspection

After cooling to room temperature, the return plates must be visually inspected for protein residues and other contamination, paying particular attention to cavities, blocks, inclusions, pipes, and other inaccessible areas. Return plates which are not free of residues must be returned for a complete retreatment process.

To ensure that neutral electrodes can be used according to their intended purpose after reprocessing, it is necessary to carry out a functional test after cleaning, disinfecting and drying the visual inspection and care measures. Carry out functional tests described in item 10.1.

Return plates with stains, which are blunt, bent, no longer function or which are otherwise damaged must be segregated!

### 10.1 Functional check

A newly purchased product must be subjected to a thorough visual and function check after its delivery and before each use. If a new product has externally visible defects or does not operate according to the instructions, contact the manufacturer or his responsible representative immediately.

When used as intended, these products are more or less subject to wear, depending on the intensity of use. This kind of wear is caused by technical factors and cannot be avoided. Therefore carry out a thorough visual examination and a function test prior to each application of these products. Carry out all plug connections only with the compatible articles and plug in the plug completely.

Products must be checked for irregularities. Paying attention to cracks, fractures and the occurrence of corrosion. Carry out a function check in accordance with the intended application of the product.

Defective products must not be used and must have undergone the complete preparatory treatment process again before being returned.

## 11 Sterilization

Prior to sterilization, products must undergo cleaning and disinfection, be rinsed off without residue using demineralized water and subsequently dried. HEBUmedical recommends using a validated steam sterilization process (e.g. sterilizer in compliance with EN 285 and validated in accordance with DIN EN ISO 17665-1).

On using the fractionated vacuum method, **sterilization** must be performed with at least **134°C (USA 132° C)** with a **minimum dwell period of 3 minutes**. Vacuum drying must then be carried out for at least 20 minutes.

<b>V Parameters used for the validation of steam sterilization</b>	
<b>Prevacuum</b>	3 times
<b>Sterilization temperature</b>	132 °C
<b>Sterilization time</b>	1,5 minutes (half cycle method)
<b>Drying time</b>	20 minutes

The vapour must be free of ingredients, recommended limiting values of feed water and vapour condensate are determined through EN 285.

Other sterilization processes are compatible but not validated from HEBUmedical.

When loading, observe the recommended total weight. After the sterilization, check the sterile product packaging for damage, and inspect the sterilization indicators.

The return plates must not be sterilized in hot-air sterilizers.

## 11.1 Packaging

Compliant packaging of products for sterilization in line with ISO 11607. Packaging used must be suitable for the instruments and protect them from microbiological contamination during storage. The seal must not be under tension. HEBUmedical recommends container or hospital common sterilization paper/film packagings for sterilized packaging.

During validation the instruments were packaged in hospital common sterilization packagings (paper/film packagings) and steam sterilized.

## 12 Lifetime

The steam sterilization procedure was validated by laboratory tests. The products were sterile validated at a pre-vacuum of at least 5min duration and a temperature of 134°C for a lifetime of 50 cycles.

You can continue to use the instruments at your own responsibility over this cycle value if the tests described in chapter 7 have been successfully completed.

## 13 Storage

Storage of the sterilized instruments in a dry, clean and dust-free environment. Products must be protected from direct sunlight in storage.

## 14 Warranty/Repair

Our products are made of high-quality materials and are carefully checked before delivery. However, even when used as intended, they are subject to wear to a greater or lesser extent depending on the intensity of use.

This wear is due to technical reasons and is unavoidable.

However, if faults occur that are not related to wear, please contact our customer service department. Defective products may no longer be used and must have undergone the entire preparatory process before being returned.

## 15 Servicing and manufacturer address

Should you require the instructions for use in paper form, please use the contact details below. The instructions for use in paper form will be made available to you within seven calendar days of receipt of the request.

Alternatively, you can print out the electronic instructions for use yourself.

HEBUmedical GmbH  
Badstraße 8  
78532 Tuttlingen / Germany  
Tel. +49 7461 94 71 - 0  
Fax +49 7461 94 71 - 22  
eMail: [service@HEBUmedical.de](mailto:service@HEBUmedical.de)  
Web: [www.HEBUmedical.de](http://www.HEBUmedical.de)

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**Our vast range of products covers over 10 000 various instruments.  
They can be found in Internet or request our catalog.**

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