## Schiebeschafftinstrumente /

Sliding shaft instruments / Instruments à tige coulissante / Instrumentos de eje deslizante / Strumenti ad albero scorrevole / Инструменти с плъзгащ се вал / Instrumenter med glidende aksel / Libisevõlli instrumendid / Liukuakselivälineet / Όργανα με συρόμενο άξονα / Instrumenti s kliznom osovinom / Bīdāmās vārpstas instrumenti / Stumdomojo veleno instrumentai / Schuifas-instrumenten / Przyrządy z wałem przesuwnym / Instrumentos de eixo deslizante / Instrumente cu arbore glisant / Instrument med glidande axel / Posuvné hriadeľové nástroje / Instrumenti z drsno gredjo / Přístroje s posuvnou hřídelí / Csúszótengelyes műszerek

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Seite 5

### **Operating Manual**

Page 21

### **Mode d'emploi**

Page 37

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Página 55

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Página 71

DEUTSCH

ENGLISH

FRANÇAIS

ESPAÑOL

ITALIANO

---

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# 1 Symbol descriptions

Symbols	Definition
	CE-labelling
	Attention
	Validated Parameters
	Manufacturer
	Lot-description
	Reference code
	Medical device / FDA Prescription device
	Medical device
	Non sterile
	Keep away from sunlight
	Dry storage required
	(Electronic) instruction for use

## 2 Introduction

By purchasing this instrument, you are now the owner of a high-quality product whose use and correct handling are described in the following.

In order to minimize possible risks to patients and users, please observe these instructions carefully. Use, disinfection, cleaning and sterilization may only be performed by suitably trained specialist personnel.

Our products are exclusively intended for professional use by appropriately trained and qualified personnel and may only be acquired by them.

## 3 Intended use

### 3.1 Intended purpose

Rectal Biopsy Specimen Forceps; Biopsy Punches	A surgical instrument for the general removal of biopsies of tumours and other tissues during surgical intervention for histopathologic investigation. It is typically made of high quality stainless steel and it is a long, inelastic instrument whose distal ends are furnished with two formed cutting shells with sharpened edges, one of them fits exactly in the other one, or scissor-like shells with a raw edge, which offer the cut of the biopsy. These ones are controlled due to a scissor-like ring handle on the proximal end. It is inserted over a man-made or a natural body opening in body cavities. The instrument is available in a large number of sizes. It is a reusable instrument intended for transient use. It is not intended for use in direct contact with the heart, central circulatory system, or central nervous system.
Intervertebral disc rongeurs; intervertebral disc punches; Laminectomy Rongeurs	A surgical hand instrument inserted between the vertebral bodies to cut off a wide, flattened slice of fibrocartilage. The instrument is usually robust to be strong enough to cut such tissue, but also thin enough to be inserted into this narrow environment. It is a reusable instrument intended for transient use. It is not intended for use in direct contact with the heart, central circulatory system, or central nervous system.
Plaster tear-off pliers; Plaster scissors	A large, scissor-type, hand-held instrument whose leaves, distal to the end, merge into various mouth shapes suitable for cutting thick layers of plaster or synthetic material used for a plaster cast on various body parts. The tip of one of the distal blades of the scissor-type cutting mechanism is typically rounded at its lower edge to prevent injury to the patient when cutting the cast. The product is usually made of stainless steel and usually has a spring mechanism that helps to open the mouth. It is a non-invasive, reusable instrument.
Tendon seizing forceps	A surgical instrument with a connected handle and two branches, usually serrated, for crossing, grasping, performing, holding or approaching sinews during surgery. It is a reusable instrument intended for transient use.

<p>Splinter Forceps; Dressing (Polypus) Forceps; Tissue grasping Forceps with tungsten carbide; Muscle grasping forceps; Intestinal and Tissue Grasping Forceps; Capsule Forceps; Prostatic grasping Forceps</p>	<p>A surgical instrument for atraumatically holding / gripping and / or squeezing intestinal tissue, tissue and some organs during a surgical procedure. It typically has two designs: 1) a self-holding, scissor-like design with ring handles; or 2) a large tweezer-like design with straight branches interlocking at the proximal end. It is available in different sizes. The working end may be made in various designs, e.g. ring-shaped, triangular, with parallel grooving or tothing to give grip, or inwardly bent profiles for grasping. The branches are typically wide and thin. It is made of stainless steel. It is a reusable instrument intended for transient use. It is not intended for use in direct contact with the heart, central circulatory system, or central nervous system.</p>
<p>Grasping Forceps</p>	<p>A surgical hand instrument consisting of two branches, which merge into two mouth parts over an end. The mouth parts are the actual working part of the forceps, which are used in different surgical treatments, therefore they are available in different shapes. It is a reusable instrument intended for transient use. It is not intended for use in direct contact with the heart, central circulatory system, or central nervous system.</p>
<p>Grasping instruments</p>	<p>An instrument for grasping and removing IUDs. It is a reusable, invasive instrument with respect to body orifices, which are not intended for connection to an active device and for transient use.</p>
<p>Goiter forceps; throat forceps; pliers (tube shaft); laryngeal polyp forceps; nose and ear tweezers; nasal septum forceps; nose pliers; Ear forceps polyp forceps; Sharp nose pliers; septum forceps</p>	<p>A general surgical instrument for transient gripping, holding or manipulating anatomical structures during an surgical ENT procedure at e.g. the bronchi, esophagus, trachea, larynx, pharynx, nose, or ear. It has a scissor-type design with ring handles, a tube shaft and a working end with different mouth designs, e.g. straight, angled, or curved with teeth or ridges to improve retention. It is made of stainless steel. It is a reusable instrument intended for transient use</p>
<p>Sphenoid Punches; Ethmoid Forceps; Antrum Punch</p>	<p>A surgical instrument for the removal of tissue or bone samples during ear, nose and throat (ENT) procedures. The instrument can be equipped with a ring handle or pistol grip. The ring handles or pistol grips have extended shafts provided with punch-like or mechanical locking mechanisms at the distal end. It is a reusable instrument intended for transient use.</p>
<p>Tonsillectomes</p>	<p>A surgical instrument used to remove the tonsils during a tonsillectomy. It can be designed either like a flexible loop to grasp and hold the almonds or like a metal housing that holds and guides a sliding knife blade (a guillotine). The flexible loop holds the tonsil so it can be removed with a surgical incision. The metal housing is available in different versions of manual cranks that guide the knife blade that cuts the almond. It is a reusable instrument intended for transient use.</p>
<p>Epiglottis Retractor; Foreign Body Levers</p>	<p>An instrument for gripping, holding or manipulating anatomical structures during an ENT procedure at e.g. the bronchi, esophagus, trachea, larynx, pharynx, nose, or ear. It is made of stainless steel. It is a reusable, invasive instrument with respect to body orifices, which are not intended for connection to an active device and for transient use.</p>

## 3.2 Indication

Sliding shaft instruments are used in the following areas:

Plaster tear-off pliers; Plaster scissors	The instrument is used for treatments. It may only be used by trained and qualified professionals.
Grasping instruments	The instrument is used for treatments in gynecology. The procedure may only be carried out by suitably trained and qualified specialist personnel.
Epiglottis Retractor; Foreign Body Levers	The instrument is used for ENT examinations and treatments. It may only be used by trained and qualified professionals.
Intervertebral disc rongeurs; intervertebral disc punches; Laminectomy Rongeurs	The instrument is used for surgical procedures on the spine. It is not intended for use in direct contact with the heart, central circulatory system, or central nervous system. The procedure may only be carried out by suitably trained and qualified specialist personnel.
Tendon seizing forceps	The instrument is used in various surgical procedures. The procedure may only be carried out by suitably trained and qualified specialist personnel.
Rectal Biopsy Specimen Forceps; Biopsy Punches	The instrument is used in various procedures for histopathological examinations. It is not intended for use in direct contact with the heart, central circulatory system, or central nervous system. The procedure may only be carried out by suitably trained and qualified specialist personnel.
Splinter Forceps; Dressing (Polypus) Forceps; Tissue grasping Forceps with tungsten carbide; Muscle grasping forceps; Intestinal and Tissue	The instrument is used in various surgical procedures. It is not intended for use in direct contact with the heart, central circulatory system, or central nervous system. The procedure may only be carried out by suitably trained and qualified specialist personnel.
Grasping Forceps; Capsule Forceps; Prostatic grasping Forceps; Grasping Forceps	
Goiter forceps; throat forceps; pliers (tube shaft); laryngeal polyp forceps; nose and ear tweezers; nasal septum forceps; nose pliers; Ear forceps	The instrument is used during surgical ENT procedures. The procedure may only be carried out by suitably trained and qualified specialist personnel.
polyp forceps; Sharp nose pliers; septum forceps; Sphenoid Punches; Ethmoid Forceps; Antrum Punch; Tonsillectomes	

## 3.3 Contraindications

### 3.3.1 General Contraindications

No general contraindications are known.

### 3.3.2 Specific Contraindications

Rongeurs are contraindicated in patients with severe spinal cord stenosis.

### 3.4 Patient target group

The products are suitable for all patient groups.

## 4 Warnings

	Medical products are delivered in a non-sterile condition and must be cleaned, disinfected and sterilized prior to their initial use.
	The use of faulty instruments is in principle forbidden and they have to go through the whole cleaning process before return.
	Please take into consideration that through higher power a bigger damage of the tissue can result: f.e. on forceps: the power at the end of the jaw is higher than at the tip of the jaw
	Remove all protective sleeves and films prior to first using or preparation for use.
	The safe combination of different products or of products with implants must be reviewed prior to clinical application by the user.
	Avoid improper throwing or dropping of instruments.
	Avoid mechanical overstressing of the instrument beyond the structural design, this can lead to breakage and deformation!
	A visual inspection of the instrument for damage and contamination must be carried out before each use!
	To prevent all contact corrosion, instruments with damaged surfaces must be separated immediately.
	If the products are used on patients with transmissible spongiform encephalopathy or HIV infection, we decline any responsibility for their reuse.
	After ophthalmical use, please pay attention to water quality during treatment (according to the specifications of AAMI TIR34 and the recommendations of the Robert-Koch-Institute on preparation of medical devices)!
	Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## 5 Handling


The type of treatment must be determined in each individual case by the surgeon in cooperation with the internist and the anaesthetist.

For operational use in various surgical disciplines must be done by appropriately trained and qualified personnel.

## 6 Opening and assembling


### 6.1 Opening

1. Close handle.
2. Press button and keep it pressed.
3. Open handle.
4. Instrument is unlocked.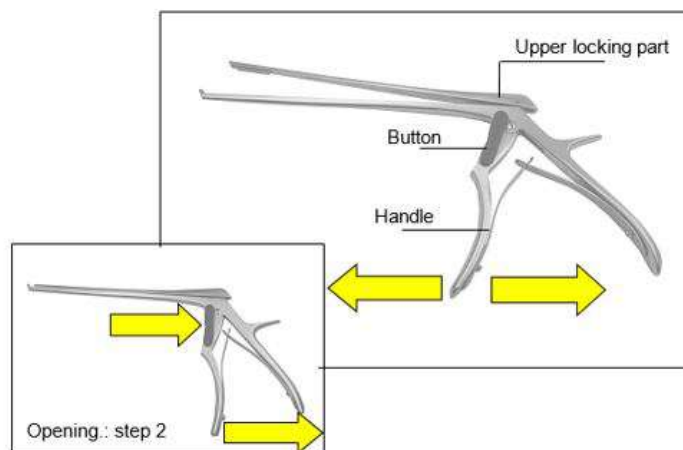

### 6.2 Assembling

1. Push down upper locking part into the guide.
2. Close the handle.
3. Instrument is locked.

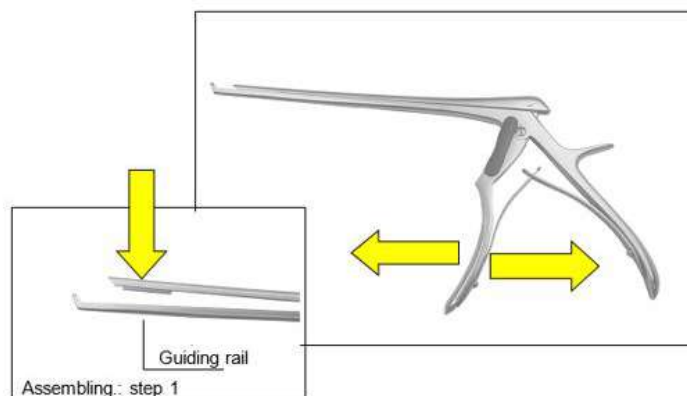## 7 Preparation

The person in charge of preparatory treatment is responsible for ensuring that the treatment is duly carried out using the relevant equipment, materials and personnel in the treatment facility and so achieves the desired result. This necessitates validation and routine monitoring of the process used. We urge you to take note of the national regulations dealing with instrument preparation.

The validated parameters refer to reusable surgical instruments. The validated parameters should also be observed for the other products described, unless a different procedure is explicitly described.

### 7.1 Information on instrument preparation

- Use cleaning and/or disinfection agents with a pH-value within 9-10.
- Please observe manufacturer instructions regarding dosage, exposure time and renewal of solutions.
- Use suitable brushes that do not remove material, i.e. **no** hard brushes (such as metal brushes and metal sponges) or coarse abrasive cleaners.
- Never leave instruments in cleaning or disinfection agents for longer than the specified time.
- Rinse and dry carefully through channels and pipes.
- Sensitive instruments must be cleaned in a storage or clamping fixture.
- Observe manufacturer instructions of cleaning – and sterilizing equipment.

#### 7.1.1 Water quality

For cleaning, neutralization and rinsing, we recommend the use of demineralized water in accordance with the "Guideline DGKH, DGSV, AKI for the validation and routine monitoring of automated cleaning and thermal disinfection processes for medical devices and on the principles of device selection" (the guideline refers to DIN EN ISO 15883-1 point 6.4.2).

According to the guideline, the following values are recommended:

- Conductivity:  $\leq 15 \mu\text{s}/\text{cm}$  (deviating from the table in DIN EN 285)
- pH value: 5 - 7
- Total hardness:  $\leq 0.02 \text{ mmol CaO}/\text{l}$
- Salt content:  $\leq 10 \text{ mg}/\text{l}$
- Phosphate (as  $\text{P}_2\text{O}_5$ ):  $\leq 0.5 \text{ mg}/\text{l}$
- Silicate (as  $\text{SiO}_2$ ):  $\leq 1 \text{ mg}/\text{l}$
- Chloride:  $\leq 2 \text{ mg}/\text{l}$

### 7.2 Preparation at the place of use

Immediately after use, remove coarse dirt from the instruments with a suitable brush (see chapter 7.1) and rinse out the working cannulas. Do not use fixing agents or hot water ( $> 40^\circ\text{C}$ ), as this results in residues becoming fixed and can affect the success of the subsequent cleaning operation

Dismantle and/or open instruments as far as possible. Within short time after use the instru-

ments clean the instruments for reducing a drying of the residues. This enables an easier cleaning. If instruments come into contact with corroding medicines or cleaning agents, wash these up with water immediately after use. Longer drying times, e.g. for dry disposal are not validated and therefore not recommended.

**V** The drying time during validation was 1 hour.

### 7.3 Ultrasound bath (optional)

All instruments must be opened, dismantled and any cavities rinsed through. Place instruments in the screen basket in such a way that overlaps and contact between instruments are avoided. Add cleaning agent to the water and adjust the temperature of the solution in line with the cleaning agent manufacturer's instructions.

The cleaning in the ultrasound bath should be at **35-40 kHz, 5 minutes** at least.

**V** To validate cleaning in an ultrasonic bath, the test items were ultrasonically treated in Neodisher Mediclean forte 0,5 % for 5 minutes.

Subsequently rinse instruments including all cavities before cleaning and disinfection. The ultrasonic bath should not be used for medical devices where material could be removed in the ultrasonic bath, e.g. products made of soft plastic or coated instruments.

### 7.4 Manual cleaning

Since mechanical processes can be standardized, reproduced and therefore validated, mechanical cleaning/disinfection should be preferred to manual processes. Manual cleaning and disinfection process is not validated and therefore needs to be validated additionally by the end user.


### 7.5 Mechanical cleaning

Due to international standards (EN ISO 15883) and national guidelines, only validated mechanical cleaning and disinfection procedures should be used. For automated cleaning, we recommend a standard program for surgical instruments, e.g. instruments from Miele.



Observe the following by loading:

- Place the disassembled or opened instruments securely in the tray.
- Instruments with openings and recesses must be placed with the open side facing downwards so that they can be cleaned and no water from the cleaning process can collect in them.

- If available, use a coordinated rinsing device.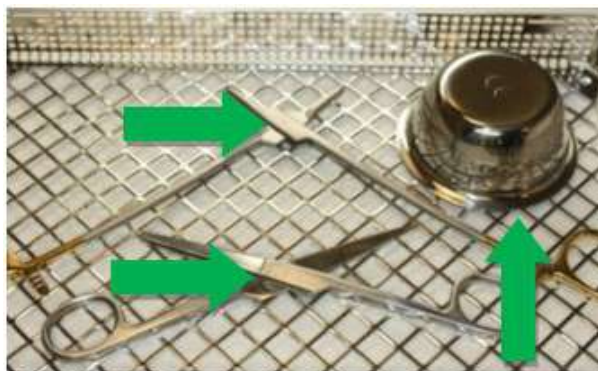

- Do not overload trays, avoid creating any overlaps

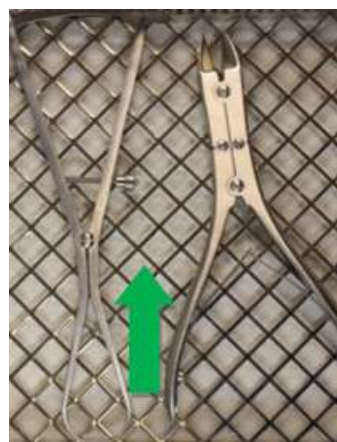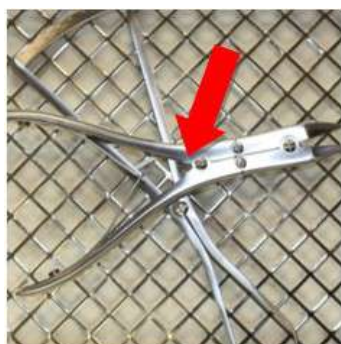Preliminary rinsing is followed by chemical.  
The chemical cleaning should take place at **40°C -60°C for at least 5 minutes.**

We recommend products with a **pH-value within 9-10**, e.g. Neodisher MediClean forte from Dr. Weigert. The cleaning agents used should be selected depending on the material and properties of the instruments and in accordance to national regulations: If there is a high chloride concentration in the water, pitting and tension crack corrosion can occur on the instruments. The occurrence of this type of corrosion is minimized by using alkaline cleaning agents and demineralized water. By adding an acid-based neutralization medium, the rinsing off of alkaline cleaning agent residues is facilitated during the first intermediate rinsing process.

In order to prevent the formation of deposits, it is advisable to use neutral cleaners where the

water quality is unfavourable.

After the second intermediate rinsing process, thermal disinfection takes place.

The thermal disinfection should be carried out with demineralized water that corresponds to microbiological drinking water quality (< 100 CFU/ml drinking water) at **80 to 95°C** and an exposure time in accordance with **EN ISO 15883**.

After the finishing of the programme take the good out of the machine because corrosion can arise if the instrument remains in the machine.

<b>V Parameters used for the validation of preparation</b>	
<b>Pre-rinsing</b>	1 minute with cold tap water
<b>Cleaning</b>	Temperature: 55 °C
	Soaking Time: 5 minutes (worst case)
	Neodischer Mediclean forte 0,4% (worst case)
<b>Neutralization</b>	Temperature: cold DI water
	Soaking Time: 2 minutes
	Neodisher Z 0,1%
<b>Post-rinsing</b>	2 minutes with cold DI water
<b>Disinfection</b>	Temperature: 90 °C (A <sub>0</sub> 3000)
	Soaking Time: 5 minutes

## 7.6 Drying

Ensure adequate drying by the cleaning and disinfection device or using other suitable measures.

<b>V</b> Drying was omitted in the validation (worst case condition).
---

## 8 Maintenance, inspection

After cooling to room temperature, the instruments must be visually inspected for protein residues and other contamination. Slits, barriers, locks, tubes and other areas that are difficult to access must be thoroughly inspected. Instruments that are not residue-free must be repeatedly subjected to the entire reprocessing process.

To ensure that surgical instruments can be used for their intended purpose after reprocessing, it is necessary to carry out a functional test after cleaning, disinfection and drying of the visual inspection and care measures. Carry out the functional tests described in chapter 8.1.

Instruments that are stained, blunt, bent, no longer functional, damaged in any other way or the instrument labeling is no longer recognizable (e.g. labeling, UDI code no longer legible) must be discarded!

To help identify faulty instruments that need to be sorted out, we recommend the brochure "Instrument Reprocessing" from the Working Group "Instrumenten Aufbereitung". This includes Chapter 8 "Checks and Care" and Chapter 12 "Surface Changes: Deposits, Discoloration, Corrosion, Aging, Swelling and Stress Cracks".

### 8.1 Functional check

A newly purchased product must be subjected to a thorough visual and function check after its delivery and before each use.

Products must be checked for irregularities. Paying attention to cracks, fractures and the occurrence of corrosion.

If there are joints, the instruments should be oiled with a care product before the functional test. We recommend a medical white oil based on paraffin oil.

Check instruments with joints for ease of movement.

The labeling must also be checked for legibility.

Carry out a function check in accordance with the intended application of the instrument.

Essential tests for instruments with serration and jaw are among others:

- Correct opening and closing (smooth movement, complete)
- Intact serration (bent, broken off)
- The jaws must close properly
- No throwing the jaws over (cross bite)

Defective products must not be used and must have undergone the complete preparatory treatment process again before being returned.

## 9 Sterilization

Prior to sterilization, products must undergo cleaning and disinfection, be rinsed off without residue using demineralized water and subsequently dried. HEBUmedical recommends using a validated steam sterilization process (e.g. sterilizer in compliance with EN 285 and validated in accordance with DIN EN ISO 17665-1).

The validated parameters refer to reusable surgical instruments. The validated parameters should also be observed for the other products described, unless a different procedure is explicitly described.

On using the fractionated vacuum method, **sterilization** must be performed with at least **134°C (USA 132°C)** with a **minimum dwell period of 3 minutes**. Vacuum drying must then be carried out for at least 20 minutes.

<b>V Parameters used for the validation of steam sterilization</b>	
<b>Prevacuum</b>	3 times
<b>Sterilization temperature</b>	132 °C
<b>Sterilization time</b>	1,5 minutes (half cycle method)
<b>Drying time</b>	Drying time

The vapour must be free of ingredients, recommended limiting values of feed water and vapour condensate are determined through EN 285.

Other sterilization processes are compatible but not validated from HEBUmedical.

When loading, observe the recommended total weight. After the sterilization, check the sterile product packaging for damage, and inspect the sterilization indicators.

### 9.1 Packaging

Compliant packaging of products for sterilization in line with ISO 11607. Packaging used must be suitable for the instruments and protect them from microbiological contamination during storage. The seal must not be under tension. HEBUmedical recommends container or hospital common sterilization paper/film packagings for sterilized packaging.

**V** During validation the instruments were packaged in hospital common sterilization packagings (paper/film packagings) and steam sterilized.

## 10 Lifetime

The steam sterilization procedure was validated by laboratory tests. The products were sterile validated at a pre-vacuum of at least 5min duration and a temperature of 134°C for a lifetime of 50 cycles.

You can continue to use the instruments at your own responsibility over this cycle value if the tests described in chapter 7 have been successfully completed.

## 11 Storage

Store products in a dry, clean and dust-free environment at moderate temperatures from 5°C to 40°C.

Protect from the effects of the sun's rays and artificial light.

## 12 Warranty / Repair

Our products are manufactured from high-grade materials and carefully checked prior to dispatch. However, even if used properly in accordance with their intended purpose they are subject to a greater or lesser degree of wear depending on their intensity of use.

This wear is technically induced and unavoidable.

Should faults occur independently of wear, please contact our customer services. Defective products should no longer be used.

They must undergo the complete preparatory treatment process before being returned.

## 13 Disposal

Before disposal, the product must undergo the complete reprocessing and sterilization process in accordance with the procedure described in these instructions for use.

**Risk of infection from contaminated products!**

- When disposing of the product, its components and their packaging, the national regulations must be observed.

**Risk of injury from sharp-edged and/or pointed products!**

- When disposing of the product, ensure that the packaging prevents injury from the product.

## 14 Service and manufacturer address

Should you require the instructions for use in paper form, please use the contact details below. The instructions for use in paper form will be made available to you within seven calendar days of receipt of the request.

Alternatively, you can print out the electronic instructions for use yourself.

HEBUmedical GmbH  
Badstraße 8  
78532 Tuttlingen / Germany  
Tel. +49 7461 94 71 - 0  
Fax +49 7461 94 71 - 22  
eMail: [service@HEBUmedical.de](mailto:service@HEBUmedical.de)  
Website: [www.HEBUmedical.de](http://www.HEBUmedical.de)

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