HB 8881

HB 8881V

HEBU Staubsauger

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Seite 3

DEUTSCH

Operating and service manual

Page 21

ENGLISH

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Page 39

FRANÇAIS

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HEBU Vacuum Cleaner

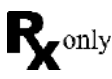
Contents

1	Symbol descriptions	23
2	Scope of delivery	23
3	Safety instructions	24
4	Scope	25
4.1	Inteded use	25
4.2	Contraindications	25
5	Operation	25
5.1	Manual mode	25
5.2	Automatic mode	26
6	Cleaning and maintenance	27
6.1	Changing the filter	27
6.2	Exchanging the exhaust filter	28
6.3	Exchanging the motor protection filter	28
6.4	Troubleshooting	29
7	Accessories/Spare parts	29
8	Vacuum cleaner spares	30
8.1	Housing spares	30
8.2	Suction turbine spares	32
8.3	Container spares	33
9	Warranty	34
10	Environmental directives	34
11	Specifications	34
12	Serving and manufacturer address	35
13	Annex	36

HEBU
Vacuum Cleaner

1 Symbol descriptions

Symbol	Definition
	CE-labelling
	Attention
	Manufacturer
	Lot-description
	Reference code
	Medical device / FDA Prescription device
	Medical device
	Non sterile
	WEEE-labelling
	(Electronic) instruction for use

2 Scope of delivery

HB 8881	HEBU Vacuum Cleaner
HB8887-14	ECO nonwoven filter
HB 8886-05	Nonwoven filter bag (1 pc.)
	Operating manual

On request, we can alter this standard configuration.

Please note the wording on your invoice.

For more information, see the chapter Accessories

3 Safety instructions

Our products are exclusively intended for professional use in accordance with their intended Purpose by appropriately trained and qualified personnel and may only be acquired by them.

All safety instructions contained in this operating manual are marked with 

	Read the operating manual and the locally applicable occupational safety regulations and ensure that these are observed before using the vacuum cleaner for the first time. <i>Please ensure that this operating manual is kept in a safe place for later use and enclose it when selling or otherwise passing on the vacuum cleaner to a third party.</i>
	Intended purpose: This device is intended to be used for the extraction of fine, dry grinding dust.
	When vacuuming, the filter may never be removed.
	Do not rivet or screw any signs or symbols onto the device. This can render the protective insulation ineffective. We recommend using adhesive signs.
	Only ever use undamaged plugs and cables. Check the plug and cable at regular intervals.
	The mains voltage and voltage specification given on the device must be in agreement.
	Only use original HEBU accessories.
	Avoid damage during storage, handling, and cleaning.
	The device may not be operated in potentially explosive atmospheres
	Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

4 Scope

Dust extraction equipment is used in orthopaedics to remove gypsum dust.

4.1 Inteded use

dust extractor; Accessories: An assortment of devices used to extract the dust produced by a plaster saw during gypsum removal (e.g., pneumatic or electric gypsum saw) from the working field. It usually consists of a motorized vacuum pump, a filter and a collection container (e.g., bag or cartridge). It is usually a mobile unit and its motor can run simultaneously with the saw motor.

4.2 Contraindications


There are no known contraindications.

5 Operation

5.1 Manual mode


Mounting the filter bag:

- Disengage the suction head and remove.
- Insert the filter
- Replace the suction head and lock into position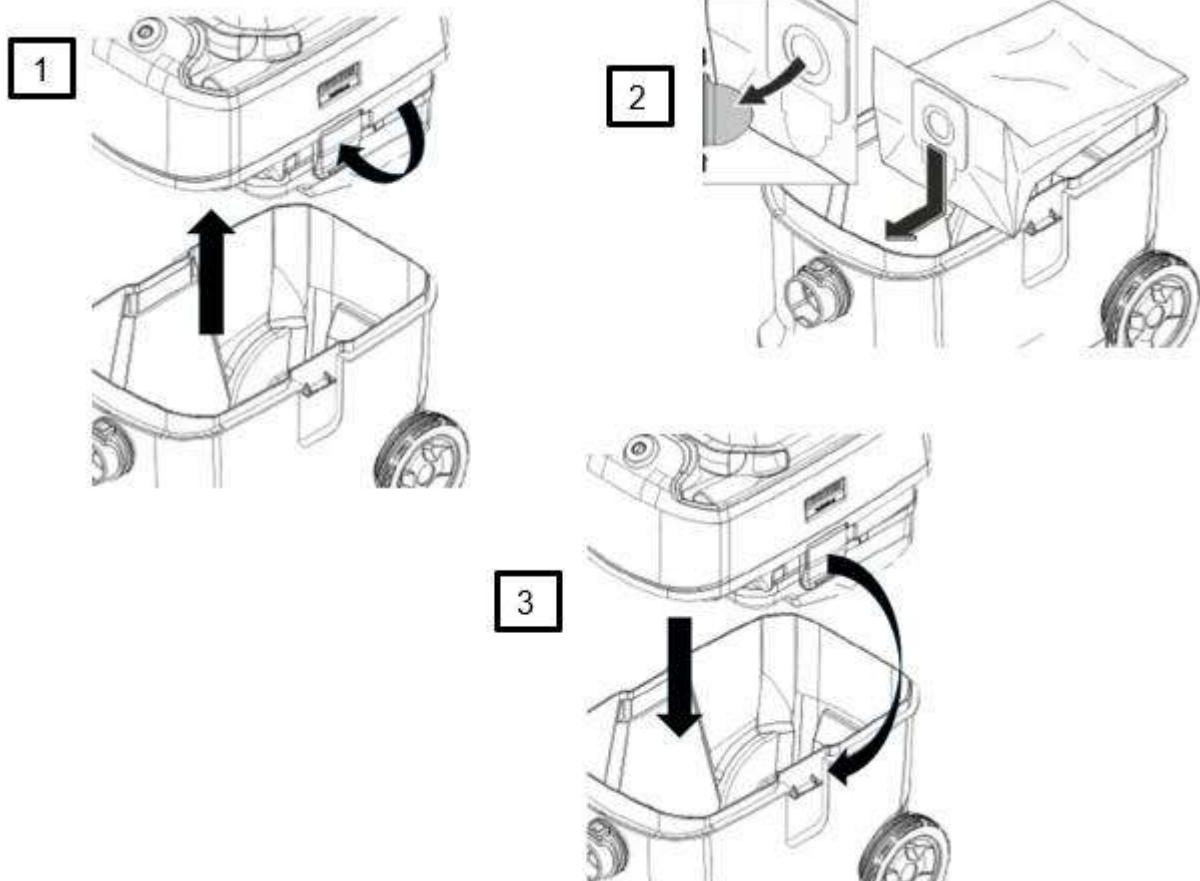


Commissioning:

- Adapt the suction hose
- Connect the mains plug.
- Switch on the device and start up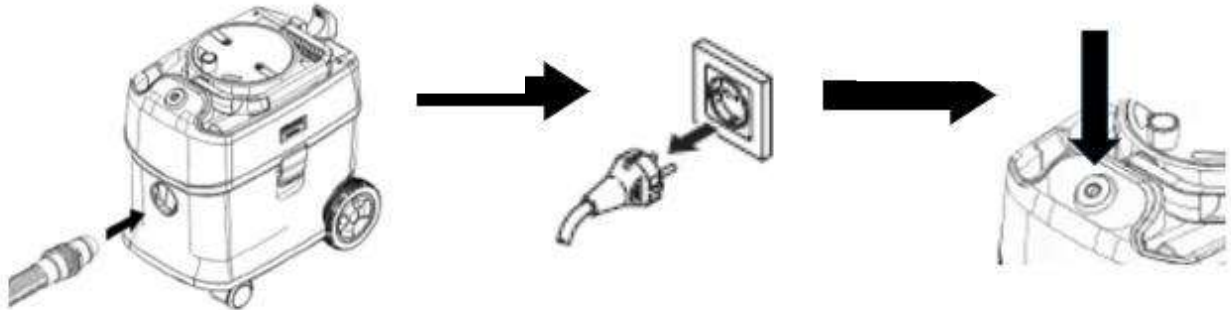

5.2 Automatic mode

- Insert the mains plug of the HEBU oscillating saw at the vacuum cleaner (1).
- Switch on the vacuum cleaner (2).
- The lamp lights up and the vacuum cleaner runs for around 0.5 seconds as a function check. The vacuum cleaner then goes to the standby mode (3)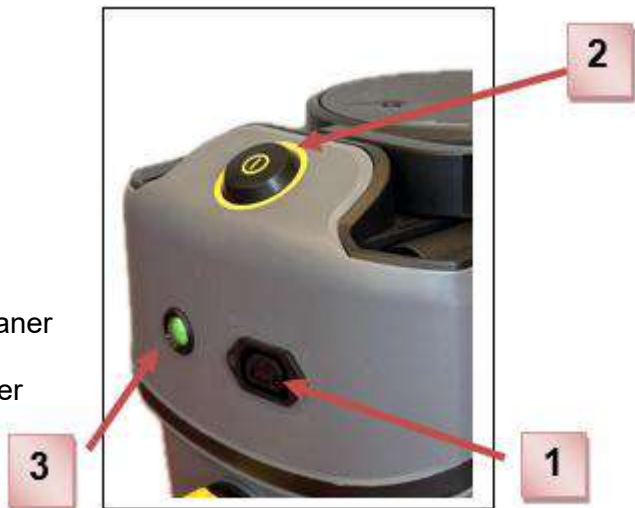

Note:

- ◆ The vacuum cleaners can be used together with a oscillating HEBU cast saw as well as the autopsy model.
- ◆ The vacuum cleaner is automatically switched on and off in the standby mode by the HEBU oscillating saw, provided you have performed the following points beforehand:
 - Every time the vacuum cleaner is first switched on, it goes to the standby mode.
 - or after being switched on again at least 6 secs. after last being switched off.
- ◆ The vacuum cleaner has a run-on time of up to 6 secs. This is intended to ensure that the vacuum cleaner hose is completely emptied.
- ◆ If the vacuum cleaner is switched on again less than 4 seconds after being switched off, the suction process operates without automatic function. However the HEBU oscillating saw can still be switched on and off.
- ◆ The voltage specification of the vacuum cleaner must agree with the voltage specification of the HEBU oscillating saw.


6 Cleaning and maintenance

	Before performing any cleaning or maintenance work, disconnect the mains plug.
	Repairs may only be performed by approved customer service centres or by specialist personnel who are familiar with all relevant safety regulations.
	Ensure that all safety checks stipulated by the relevant locally applicable regulations for mobile commercially used devices are observed.

- Exchange the motor protection filter every three months (otherwise suction loss or damage to the suction turbine can occur).
- Exchange the exhaust filter once a year.
- As cleaning agent we recommend products with a pH value between 9 and 10, e.g. MediClean forte by Dr. Weigert.


6.1 Changing the filter

- Open the filter cover
- Exchange the filter
- Close the filter cover, ensuring that it clicks audibly into place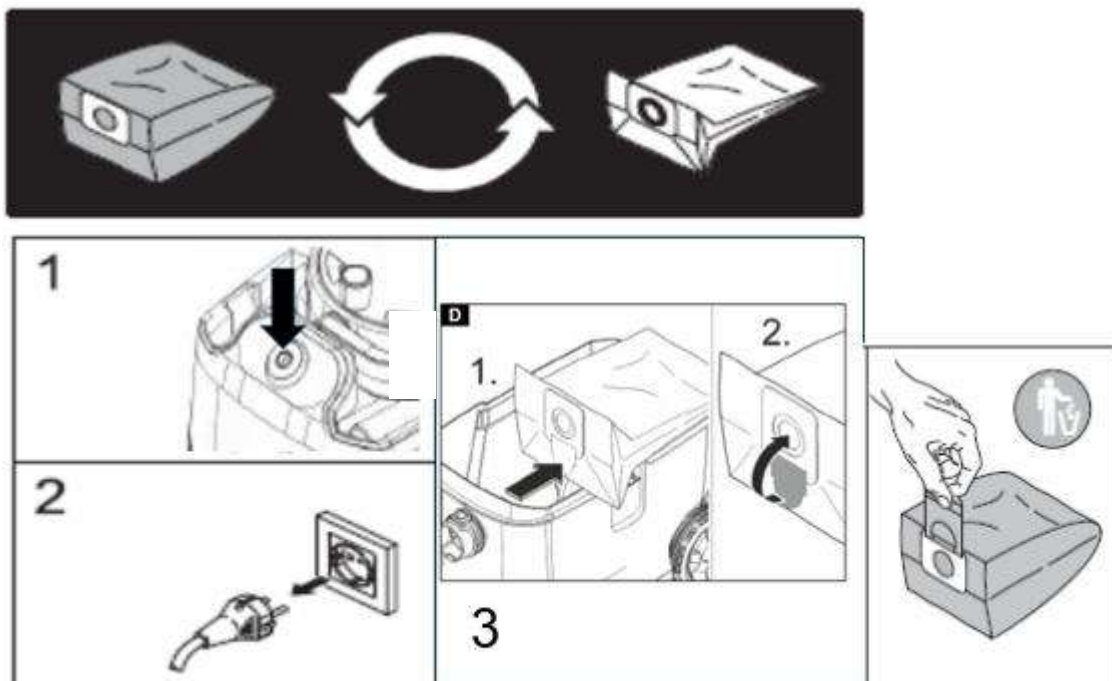


6.2 Exchanging the exhaust filter

- Leave the vacuum cleaner to cool down for around 30 minutes (if it has just been in use)
- Open the filter cover
- Exchange the exhaust filter
- Close the filter cover, ensuring that it clicks audibly into place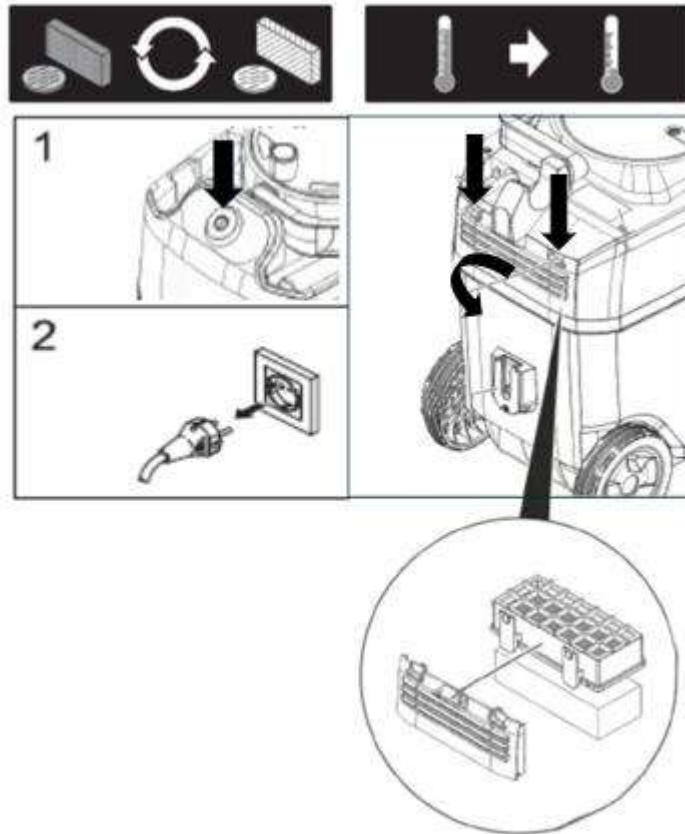

6.3 Exchanging the motor protection filter

- Leave the vacuum cleaner to cool down for around 30 minutes (if it has just been in use)
- Open the filter cover
- Remove the filter holder
- Exchange the motor protection filter
- Replace the filter holder
- Close the filter cover, ensuring that it clicks audibly into place.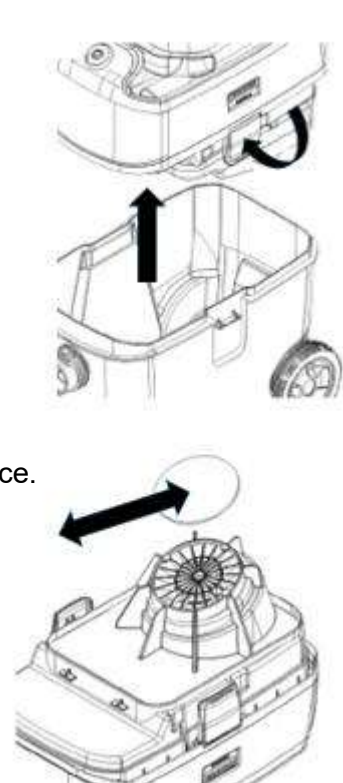

6.4 Troubleshooting

Suction turbine is not running

- Check the cable, plug, fuse and socket.
- Switch on the device

The suction turbine switches off

- Empty the container

The suction turbine does not start up again after emptying the container

- Switch off the device and wait for 5 seconds, then switch back on

Device loses suction

- Remove blockage from dust removal unit, suction hose, ventilation openings or filter
- Exchange paper filter bag/nonwoven filter
- Lock filter cover correctly into place
- Exchange motor protection filter.
- Exchange exhaust filter

Dust emerges while vacuuming

- Check correct mounting position of the filter
- Exchange the motor protection filter
- Exchange the exhaust filter

7 Accessories/Spare parts

HB8887-14	ECO nonwoven filter (Filter alone)
E8881-62	Housing for ECO nonwoven filter
HB8887-11	ECO nonwoven filter HEPA (incl.Housing)
HB8882-02	Flexible 2 m suction hose
HB8886-05	Non-woven filter bags (10 pcs.)
HB8887-12	Filter basket
HB8887-13	Motor protection filter

8 Vacuum cleaner spares

8.1 Housing spares

(drawing numbers refer to Fig. 1)

Pos.	Batch	Description	HEBU-Nr.	
			HB 8881	HB 8881V
1	1	Hood Side	E8881-51	
2	1	Spring Switch	E8881-52	
3	1	Main Switch	E8881-53	
4	1	Air Flow Hood	E8881-54	
5	1	Foam for Supply	E8881-55	
6	20	Screw 4x22xST-10.9-R2R (IN6RD)	E8881-56	
7	1	Cable Reel Complete	E8881-57	
8	1	Fuse Holder	E8881-41	
9	1	Fuse T10.0AL250V / T16.0A	E8881-19	E8881-19V
10	1	Hood Top	E8881-58	
12	1	Handle	E8881-59	
13	2	Bolt, Rotary Handle	E8881-60	
15	1	Strain Relief	E8881-61	
16	1	Housing for ECO Fleece Filter	E8881-62	
17	1	ECO Fleece Filter (mat only)	HB8887-14	
18	1	Filter Grille, Exhaust Opening	E8881-64	
19	1	Control Board, Automatic Switch-On 230V/110V	E8881-18	E8881-18V
20	2	Logo Plate	E8881-65	
21	1	Socket, Cable Saw	E8881-66	
22	1	Switch	E8881-67	
23	1	Main Switch Holder	E8881-68	
24	1	Control LED (green)	E8881-17	

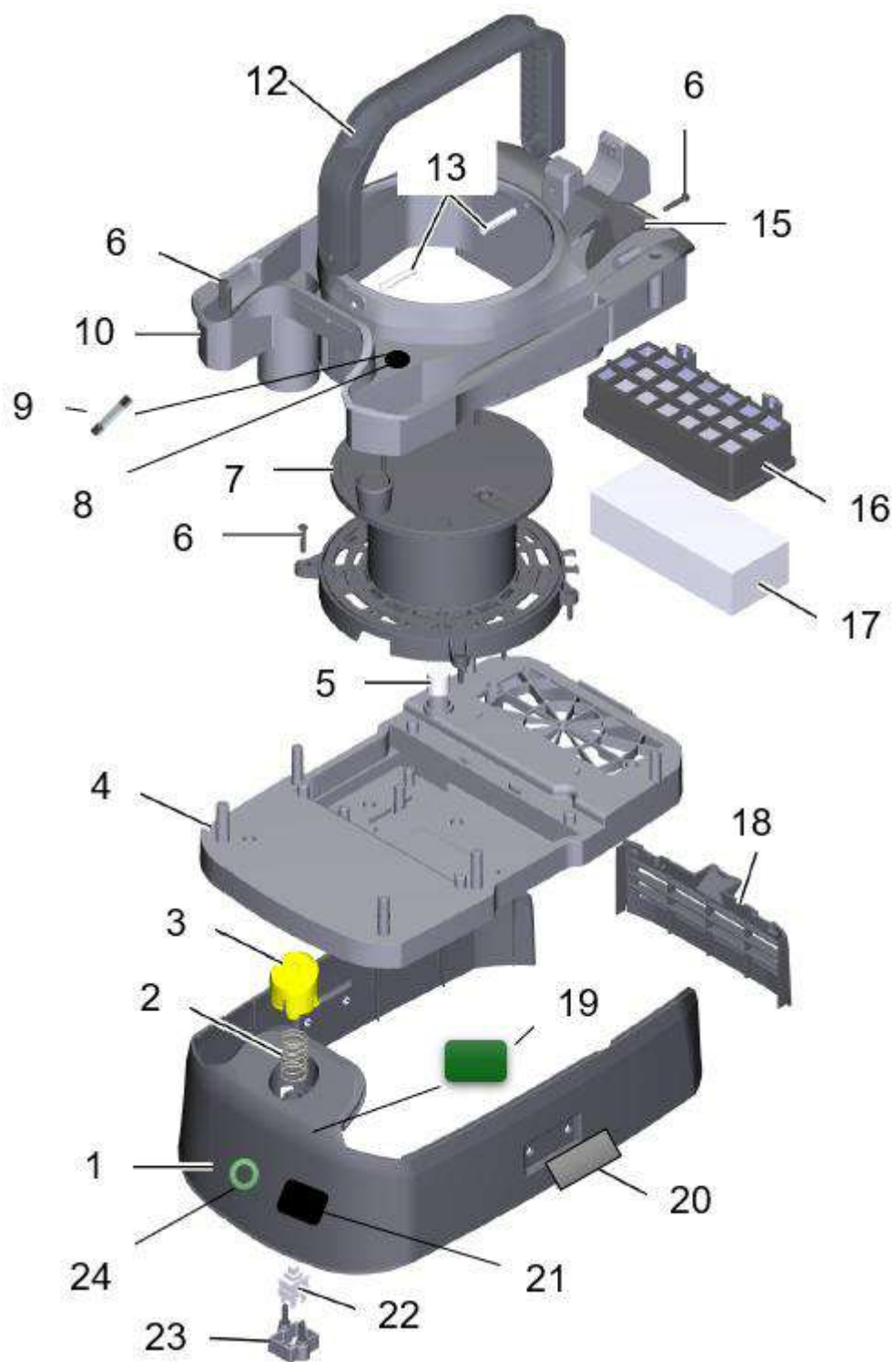
Fig. 1:
Explosion drawing of the housing

8.2 Suction turbine spares

(drawing numbers refer to Fig 2)

Pos.	Batch	Description	HEBU-Nr.	
			HB 8881	HB 8881V
1	1	Filter Motor Protection	HB8887-13	
2	1	Sealing Cord Foam Rubber	E8881-69	
3	2	Insulating Mat Set Complete For Replacement	E8881-70	
4	1	Base Plate	E8881-71	
5	1	Motor Cable	E8881-72	
6	1	Ferrite Ring	E8881-73	
7	1	Cover Air Duct Partition	E8881-74	
9	1	Motor Mount Complete For Replacement	E8881-75	
10	1	Suction Turbine	E8881-76	
12	2	Locking Tab	E8881-77	
13	1	Filter Basket Complete	HB8887-12	

Fig. 2:
Explosion drawing
the suction turbine

8.3 Container spares

(drawing numbers refer to Fig 3)

Pos.	Batch	Description	HEBU-Nr.	
			HB 8881	HB 8881V
1	1	15L container	E8881-78	
2	2	50mm casters with brakes	E 8881-50	
3	1	T suction connection	E8881-79	
4	1	Retaining ring	E8881-80	
5	1	4x22xST-10.9-R2R (IN6RD) screw	E8881-56	
6	1	Suspension adapter	E8881-81	
7	2	120mm wheel diameter	E8881-82	
8	2	Bolt	E8881-83	

Fig. 3:
Explosion drawing of the container

9 Warranty

We offer a warranty on the HEBU vacuum cleaner in compliance with statutory and country-specific regulations (presentation of invoice or delivery note required). The warranty period is at least 18 months. In the event of material or production defects, we provide repair of the device free of charge. Any attempts by users to carry out their own repairs will void any warranty claims. Damage caused as a result of incorrect treatment, overload or normal wear and tear are not covered by the warranty. Please contact our after sales service department. In the event of a warranty claim or repair, please enclose a copy of the invoice or delivery note with the device

10 Environmental directives

From the implementation date of the European directive 2002/96/EU as national legislation, the following regulations apply:

- Electrical and electronic units must not be disposed of with normal household waste.
- The consumer is required by law to return electrical and electronic devices at the end of their service life to designated public collection points or to the place of purchase.

Disposal details are set out by the applicable local legislation in the relevant country. The symbol on the product, the instructions for use or the packaging indicates the relevant applicable legislation. By sending end-of-life devices for reuse, material recycling or other forms of utilization you are making a major contribution to environmental protection. Please also send the unit packaging for environmentally responsible recycling after the end of the product's life.

11 Specifications

Model	HB 8881	HB 8881 (V)
Mains connection	220-240 Volt, 50/60 Hz.	110-120 Volt, 50/60 Hz.
Rated consumption	585 Watt	585 Watt
Vacuum	185 mbar	18,5 KPa
Container capacity	ca. 15 l	ca. 15 l
Weight	6,9 kg	6,9 kg
Air quantity	38 L/s	38 L/s
Filter performance	99,97 % > 0,5 µm	99,97 % > 0,5 µm
Safety class	II 	II 
Protection rating (IP)	IP2X	
Noise level (A-rated)	52 dB (A)	
Standards	ISO 9001:2000 und ISO 13485:2003	

12 Servicing and manufacturer address

Should you require the instructions for use in paper form, please use the contact details below. The instructions for use in paper form will be made available to you within seven calendar days of receipt of the request.

Alternatively, you can print out the electronic instructions for use yourself.

HEBUmedical GmbH
Badstraße 8
78532 Tuttlingen / Germany
Tel. +49 7461 94 71 - 0
eMail: service@HEBUmedical.de
Web: www.HEBUmedical.de

13 Annex

Complete set including saw and accessories

HB 8888-01

With saw HB 8884, vacuum cleaner + hose + hose holder + dust removal unit + saw blade 8896-03 + tools (but without HB 8883-03 and HB 8883-05)

Only optional
accessories

Accessories (optional)

HB 8883-03

Hanging kit with handle, height adjustable

**HB 8883-05**

Magnetic shelf for saw blades and tools

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