HB8888-01

HB8889-01

HEBU

Staubabsaugung Set

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HEBU Dust extractor Set

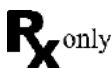
Contents

1	Symbol descriptions.....	15
2	Scope of delivery	16
2.1	Scope of delivery for complete set.....	16
2.2	Scope of delivery for re-fitting set	16
3	Safety instructions.....	17
4	Scope	17
4.1	Inteded Use	17
4.2	Contraindications	17
5	Installation	18
5.1	Installation Dust Removal Unit.....	18
5.2	Automatic mode	18
6	Cleaning and maintenance	19
7	Accessories / Spare parts	20
8	Accessories / Optional	20
9	Warranty	21
10	Environmental directives.....	21
11	Servicing and manufacturer address	22

HEBU
Dust Extractor Set

1 Symbol descriptions

Symbol	Definition
	CE-labelling
	Attention
	Manufacturer
	Lot-description
	Reference code
	Medical device / FDA Prescription device
	Medical device
	Non sterile
	WEEE-labelling
	(Electronic) instruction for use

2 Scope of delivery

2.1 Scope of delivery for complete set

HB 8881	HEBU Vaccum Cleaner
HB 8887-14	ECO nonwoven filter
HB 8886-05	Nonwoven filter bag (1 pc.)
HB 8882-02	Tube complete
HB 8884	Complete with saw TITANIUM
HB 8896-03	Saw Blade
	Dust Removal Unit
	Tools
	Operating manual

2.2 Scope of delivery for re-fitting set

HB 8881	HEBU Vaccum Cleaner
HB 8887-14	ECO nonwoven filter
HB 8886-05	Nonwoven filter bag (1 pc.)
HB 8882-02	Tube complete
	Dust Removal Unit
	Tools
	Operating manual

*On request, we can alter this standard configuration.
Please note the wording on your invoice.
For more information, see the chapter Accessories*

3 Safety instructions

Our products are exclusively intended for professional use in accordance with their intended purpose by appropriately trained and qualified personnel and may only be acquired by them.

All safety instructions contained in this operating manual are marked with !.

Read the operating manual and the locally applicable occupational safety regulations and ensure that these are observed before using the vacuum cleaner for the first time. Please ensure that this operating manual is kept in a safe place for later use and enclose it when selling or otherwise passing on the vacuum cleaner to a third party.

Intended purpose: This device is intended to be used for the extraction of fine, dry grinding dust.

When vacuuming, the filter may never be removed.

Do not rivet or screw any signs or symbols onto the device. This can render the protective insulation ineffective. We recommend using adhesive signs.

Only ever use undamaged plugs and cables. Check the plug and cable at regular intervals.

The mains voltage and voltage specification given on the device must be in agreement.

Only use original HEBU accessories.

Avoid damage during storage, handling, and cleaning.

The device may not be operated in potentially explosive atmospheres

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

4 Scope

Dust extraction equipment is used in orthopedics to remove gypsum dust.

4.1 Inteded Use

dust extractor; Accessories: An assortment of devices used to extract the dust produced by a plaster saw during gypsum removal (e.g., pneumatic or electric gypsum saw) from the working field. It usually consists of a motorized vacuum pump, a filter and a collection container (e.g., bag or cartridge). It is usually a mobile unit and its motor can run simultaneously with the saw motor.

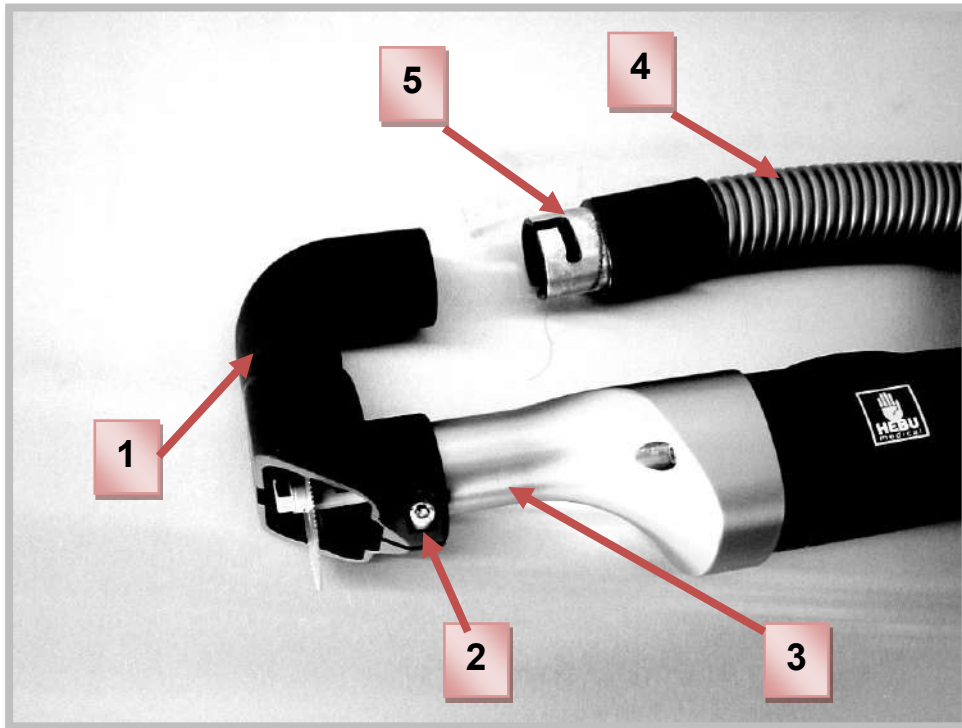
4.2 Contraindications

There are no known contraindications.

5 Installation

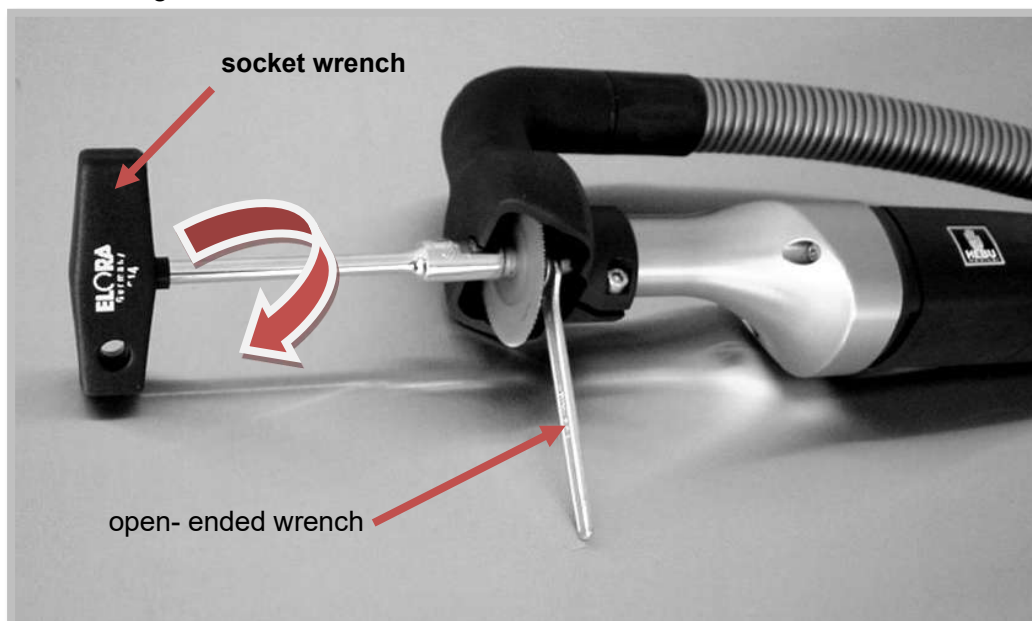
5.1 Installation Dust Removal Unit

- Pos.1 to clip on with Pos.3 and to tighten with Pos. 2.
- Pos. 5 (o- ring) to put over Pos. 4 and to grease bajonett fitting.
- Pos. 1 to connect the bajonett fitting over Pos. 4.

5.2 Automatic mode

- To draft the saw blade and manual the screw to edge upwards.
- To apply open- ended wrench on shaft to secure.
- To tighten the screw with socket wrench.


	<p>Always at the change from the saw blade, apply both wrenches! Socket wrench to attract and open-ended wrench to secure the screw!</p>
---	---

6 Cleaning and maintenance

	<p>Before performing any cleaning or maintenance work, disconnect the mains plug.</p>
	<p>Repairs may only be performed by approved customer service centres or by specialist personnel who are familiar with all relevant safety regulations.</p>
	<p>Ensure that all safety checks stipulated by the relevant locally applicable regulations for mobile commercially used devices are observed.</p>
	<p>Bajonett fitting between dust removal unit and vacuum tube must be grease with HEBU special lubrication grease.</p>
	<p>The bajonett fitting should be spaced to get cleaned and greased.</p>
	<p>The o-ring should be spaced to be under examination to form cracks and deformation.</p>

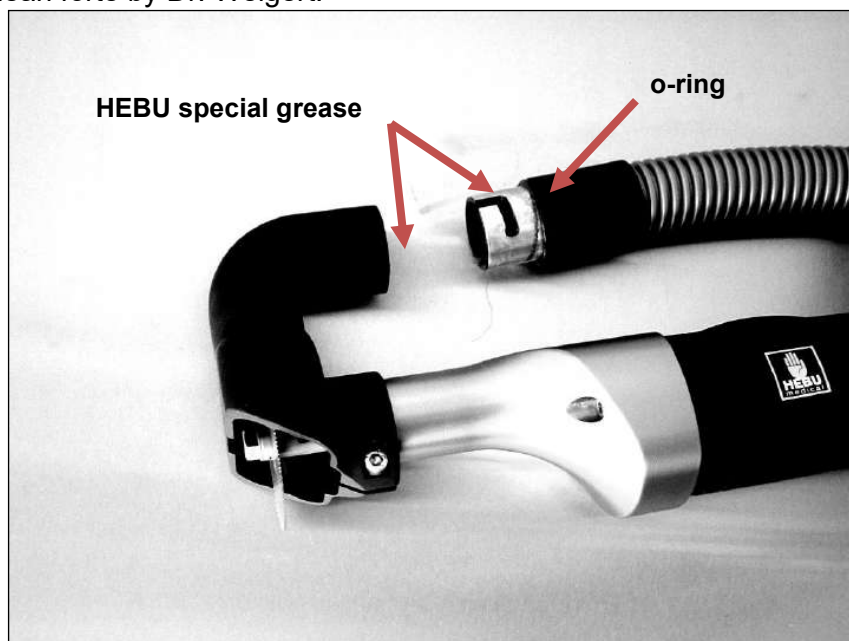- Exchange the motor protection filter every three months (otherwise suction loss or damage to the suction turbine can occur).
- Exchange the exhaust filter once a year.
- As cleaning agent, we recommend products with a pH value between 9 and 10, e.g. MediClean forte by Dr. Weigert.

7 Accessories / Spare parts

HB 8882-02	Tube complete with 2m
E 8888-01	Socket wrench with T-handel 11 mm
HB 8877	Tube with special grease
E 8894-01	Open- ended wrench 11 mm
E 8894-02	Spare screw for saw blade
E 8894-03	Fixation ring for screw

8 Accessories / Optional

HB 8883-03	Hanging kit with handle, height adjustable
-------------------	--

HB 8883-05 Magnetic shelf for saw blades and tools

9 Warranty

We offer a warranty on the HEBU Dust Extractor in compliance with statutory and country-specific regulations (presentation of invoice or delivery note required). The warranty period is at least 18 months. In the event of material or production defects, we provide repair of the device free of charge. Any attempts by users to carry out their own repairs will void any warranty claims.

Damage caused as a result of incorrect treatment, overload or normal wear and tear are not covered by the warranty. Please contact our after sales service department.

In the event of a warranty claim or repair, please enclose a copy of the invoice or delivery note with the device

10 Environmental directives

From the implementation date of the European directive 2002/96/EU as national legislation, the following regulations apply:

- Electrical and electronic units must not be disposed of with normal household waste.
- The consumer is required by law to return electrical and electronic devices at the end of their service life to designated public collection points or to the place of purchase.

Disposal details are set out by the applicable local legislation in the relevant country. The symbol on the product, the instructions for use or the packaging indicates the relevant applicable legislation. By sending end-of-life devices for reuse, material recycling or other forms of utilization you are making a major contribution to environmental protection.

Please also send the unit packaging for environmentally responsible recycling after the end of the product's life.

11 Servicing and manufacturer address

Should you require the instructions for use in paper form, please use the contact details below. The instructions for use in paper form will be made available to you within seven calendar days of receipt of the request.

Alternatively, you can print out the electronic instructions for use yourself.

HEBUmedical GmbH
Badstraße 8
78532 Tuttlingen / Germany
Tel. +49 7461 94 71 - 0
eMail: service@HEBUmedical.de
Web: www.HEBUmedical.de

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