**HB 8875-18**

# Akku, Li-Ion, 4000 mAh

**HEBU Akku / HEBU Battery**  
Für Oszillosäge / For Oscillo-Saw

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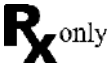
**HB 8875-18**

# Operating instructions HEBU-Battery, Li-Ion, 4000 mAh

## Contents

|     |  |    |
|-----|--|----|
| 1   | Symbol descriptions .....                | 10 |
| 2   | Safety instructions .....                | 11 |
| 3   | Scope.....                               | 11 |
| 3.1 | Intended Use.....                        | 11 |
| 3.2 | Contraindications .....                  | 11 |
| 4   | Commissioning and charging .....         | 12 |
| 4.1 | Battery charge condition .....           | 12 |
| 4.2 | LED indicator .....                      | 12 |
| 4.3 | Spontaneous discharge .....              | 12 |
| 5   | Maintenance .....                        | 13 |
| 6   | Storage .....                            | 13 |
| 7   | Environmental protection.....            | 13 |
| 8   | Specifications.....                      | 13 |
| 9   | Manufacturer and servicing address ..... | 13 |

# 1 Symbol descriptions

| Symbol  | Definition                               |
|---|--|
|    | CE-labelling                             |
|    | Attention                                |
|    | Manufacturer                             |
|    | Lot-description                          |
|    | Reference code                           |
|   | Medical device / FDA Prescription device |
|  | Medical device                           |
|  | Non sterile                              |
|  | WEEE-labelling                           |
|  | (Electronic) instruction for use         |

## 2 Safety instructions

|   |  |
|---|--|
|    | Batteries must not be disassembled, opened or reduced in size. Do not subject batteries to mechanical impact or shock. Hazardous vapours and fluid can escape in case of damage and improper use of the battery. The vapours can irritate the respiratory system. Liquid ejected from the battery may cause skin irritations or burns. At accidental contact immediately obtain medical aid. |
|    | When battery fluid from a damaged battery has come into contact with objects close by, check the respective components, clean them or replace them as required.  |
|    | Batteries are sealed and must not be opened.   |
|    | Do not burn or open by force – risk of explosion!  |
|    | Protect from moisture.   |
|    | Keep away from children.   |
|   | Before any work on the saw itself, remove the battery from the saw.  |
|  | Remove the battery only when the saw is switched off.  |

## 3 Scope

### 3.1 Intended Use

A rechargeable set of electrochemical cells or a single cell, designed to store chemical energy and release it in the form of electrical energy to provide power for active implantable medical devices or external medical instruments, as a power supply for programmable devices that keep electronic information, or to power portable or other medical devices, when it is not possible or convenient to use a cable. The size, shape and chemical composition of the battery should be specified according to the requirements of the appropriate IEC standard.

The use mainly takes place in orthopedics for plaster removal.

### 3.2 Contraindications

There are no known contraindications.

## 4 Commissioning and charging

- Charge before first-time-use.
- Charge only within the operating temperature range.

Recharge only with the charger specified by HEBU.

Use the battery only with suitable saws.

A charger which is suitable for one type of battery pack may create a risk of fire when used with another battery pack.

For charging Li-Ion batteries only use HEBU battery chargers HB 8879 or HB 8879V.

### 4.1 Battery charge condition

To check the battery charge status, press the battery button located to the left of the battery status display, as shown in figure 1. The actual percentage charge status of the battery is only displayed when the motor of the power tool is stopped.

Figure 1 Battery charge condition

### 4.2 LED indicator

| LED indicator        | Meaning                            | Activity  |
|----------------------|------------------------------------|---|
| 1-5 green LED        | Percentage of charge condition     | Operation   |
| Continuous red light | Battery is almost empty            | Charge battery  |
| Red flashing light   | Battery is not ready for operation | Bring the battery into the battery operating temperature range, then charge |

The electronics automatically switch off the motor prior to the battery being deep discharged. Operate and charge the battery only within the battery operating-temperature range of 0 °C – 45 °C (32 °F – 113 °F). At the beginning of the charging procedure, the battery temperature must be within the battery operating temperature range.

### 4.3 Spontaneous discharge

Charged batteries are subjected to spontaneous discharging at a rate which is governed by the ambient temperature. This means that if they are stored for more than two months before use, recharging will be necessary. The higher the temperature, the shorter this period. The lower the temperature, the lower the rate of spontaneous discharge.

## 5 Maintenance

Always keep clean the electrical contacts at the battery and the saw. In case you need to clean the battery, use a dry and dust - free cloth. Beyond, no maintenance is necessary.

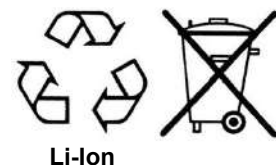
If the battery's output fades rapidly despite correct recharging, its operating lifetime is exhausted.

## 6 Storage

Store the battery at room temperature, dry, dust free and protected from sun rays. It is recommended to keep the genuine packaging of the battery, to store it in when needed.

## 7 Environmental protection

- Do not open the battery!
- Faulty batteries must be recycled in accordance with directive 91/157/EWG.
- Do not throw batteries away with the domestic waste, into fire or water.

HEBU provides environment-friendly battery disposal:

Faulty batteries are fully recycled. They can be returned to the specialist dealer or the HEBU customer service centre.

## 8 Specifications

| Model                 | HB 8875-18  |
|-----------------------|-------------|
| Voltage               | 18 V DC     |
| Rated capacity        | 4000 mAh    |
| Weight                | 0,55 kg     |
| Operating temperature | 0°C to 45°C |

## 9 Manufacturer and servicing address

Should you require the instructions for use in paper form, please use the contact details below. The instructions for use in paper form will be made available to you within seven calendar days of receipt of the request.

Alternatively, you can print out the electronic instructions for use yourself.

HEBUmedical GmbH  
 Badstraße 8  
 78532 Tuttlingen / Germany  
 Tel. +49 7461 94 71 - 0  
 eMail: [service@HEBUmedical.de](mailto:service@HEBUmedical.de)  
 Web: [www.HEBUmedical.de](http://www.HEBUmedical.de)

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**Our vast range of products covers over 10 000 various instruments.  
They can be found in Internet or request our catalog.**

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